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04/2018/JP

# Instructions for XXL-Rehab Comfort cushion

## CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 To ensure proper use of the XXL-Rehab Comfort cushion, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .

Also remember that in order to prevent the risk of injury to the user, the cushion may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

## Use

The XXL-Rehab Comfort Cushion is designed as a standard wheelchair cushion to prevent pressure relief for bariatric users with a weight of up to 325 kg.

 If users are weighing more than 325 kg there is a risk that the cushion will break or take damage.

## Description

The XXL-Rehab Comfort cushion is made of a 3 cm Visco-elastic foam on the top, and the base is made of a thick Heavy Duty PU foam with springs that cut into the under base of foam.

The cushion has a patented system of springs which distributes the pressure to a larger surface and assures pressure relief.

## Description of cover

100% polyurethane cover. It is waterproof, breathable, antiallergenic, and fireproof.

## Before use

Place the cushion on a chair/in a wheelchair with the zipper placed at the bottom.

 The XXL-Rehab Comfort cushion must always be used with the cover zipped correctly.

 Do not place any further layers or padding between the cushion and the user as this may reduce the effect of the cushion.

## During use

 Inspect the cushion on a regular basis to ensure that the cushion and the cover are properly oriented and inspect if the cushion is wet or damaged.

# Maintenance and cleaning Specifications

## Of cushion

⚠ The cushion cannot be cleaned.

## Of cover

- Machine wash at a maximum of 95° C.
- 134° C sterilisation treatment is possible (Autoclaved maximum 3 cycles).
- No bleach, Standard disinfectants may be used.
- Do not iron.
- No dry washing.

⚠ If the cover is damaged do not use the cushion before the cover is replaced.

## Garanti

The XXL-Rehab Comfort cushion is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended uses; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

Maximum weight capacity is	325 kg
Weight	Approx. 2 kg
Base cushion	HD PUR foam / Viscoelastic PUR foam
Cover	PU
Incontinence cover	Yes
Washable cover	Yes
Flame retardant cover	Yes

The PU and the Viscoelastic foam has the OEKO-TEX Standard 100 CLASS I certification which is a European Standard guaranteeing the omission of harmful substances in the composition of textile products during the entire manufacturing process until final delivery to the client.

## MANUFACTURER AND DEALER

Cobi Rehab  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
Tel.: +45 70 25 25 22  
www.cobirehab.com

04/2018/JP

# COBI REHAB<sup>®</sup>

The Positioning Company

**COBI REHAB**  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
Tel. +45 7025 2522  
[www.cobirehab.com](http://www.cobirehab.com)