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Instructions for XXL-Rehab Crutch

CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 To ensure proper use of the XXL-Rehab Crutch, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .

Also remember that in order to prevent the risk of injury to the user, the cane may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

Use

The XXL-Rehab Crutch is designed to provide support and security for a person with a weight of up to 325 kg.

 If users are weighing more than 325 kg there is a risk that the elbow crutch will break or take damage.

Description

The XXL-Rehab Crutch is produced with clips and has a continuous rod for height adjustment.

Before use

The forearm support and the hand support are height adjustable by removing the clip and placing the inner tube of the cane in the desired position.

Stick the clip through the holes and then press the clip into place.

 The XXL-Rehab Crutch may not be used before the clip for height adjustment is pressed fully in place, otherwise there is danger that inner tube might loosen at risk for the user to fall.

Maintenance and cleaning

The XXL-Rehab Crutch can be cleaned with water and ordinary detergent.

Clips and the continuous rod for height adjustment should be regularly inspected and if necessary replaced.

⚠ Failure to inspect the clip may cause malfunction.

Warranty

The XXL-Rehab Crutch is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

XXL-Rehab Crutch is manufactured and tested in accordance with international standards EN ISO 11334-1:2007.

Specifications

Maximum weight capacity is	325 kg
Height adjustable	68-110 cm
Weight	1.5 kg
Underarm support	15 cm
Grip	14 cm
Anatomical grip	Yes
Reflex	Yes
Material	Anodized steel
Grip	PU/Vinyl
ISO - classification	ISO 12 03 06 - 04

XXL-Rehab Crutch with anatomical grip.

MANUFACTURER AND DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
www.cobirehab.com

04/2018/JP

COBI REHAB[®]
The Positioning Company

COBI REHAB
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel. +45 7025 2522
www.cobirehab.com