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04/2018/JP

# Instructions for XXL-Rehab Bedside Commode

## CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 To ensure proper use of the XXL-Rehab Bedside Commode, please read these instructions thoroughly before using the product. Pay particular attention to the warning notes marked with .

Also remember that in order to prevent the risk of injury to the user, the bedside commode should be adjusted only by trained caregivers or aides or by individuals who have been instructed in its proper use.

## Before use

The bolts for adjusting the legs and backrest must be properly mounted and tightened in the holes on the side of the bedside commode.

 Do not use the bedside commode before the bolts for adjusting the legs and backrest have been properly mounted and tightened, as otherwise there is a risk of the user falling off to the side of the commode.

## In use

 To prevent the risk of tipping, only use the bedside commode on a level, hard surface.

 If the user requires assistance during transfers to the bedside commode, the aides must be particularly aware of the risk of performing unsafe lifts or movements when placing the user on the bedside commode. This applies whether the transfer is effected by means of a lift, slide board or support device.

## Use

The XXL-Rehab Bedside Commode is designed to help an individual weighing up to 325 kg during lavatory use and in the shower.

 Using the product for individuals weighing more than 325 kg may pose a risk of tipping or damaging the bedside commode.

 Always use the XXL-Rehab Bedside Commode on level, hard flooring with a maximum slope of 3 degrees, as otherwise there is a risk of tipping.

## Maintenance and cleaning

The white frame components and the seat and armrests may be cleaned with water and a household detergent. To thoroughly clean the seat: remove the seat, clean and make sure to fasten/press down the seat properly again.

Regularly check all bolts and nuts, and retighten if required.

**⚠** Failure to inspect the bolts as directed may cause functional failures.

## Warranty

The XXL-Rehab Bedside Commode is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

Maximum weight capacity is	325 kg
Total length	80-101 cm
Total outer width on armrests	73 / 83 cm
Total height	79-94 cm
Seat width	61 / 71 cm
Seat depth	40-52 cm
Seat height	44-59 cm
Height of backrest	37 cm
Width toilet opening	Diameter 24 cm, 35 cm long
Distance from floor to seat underside	33-48 cm
Distance from floor to bucket underside	21-36 cm
Total outer width of front legs	69 / 79 cm
Seat to armrest	26 cm
Weight	14 / 15 kg
Frame	Powder-lacquered chromium molybdenum steel
Seat / back / armrests	Polyurethane
ISO - classification	ISO 09 33 03

XXL-Rehab Bedside Commode is manufactured and tested in accordance with the international DS/EN 12182:2000 standard.

### MANUFACTURER AND DEALER

Cobi Rehab  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
Tel.: +45 70 25 25 22  
www.cobirehab.com

04/2018/JP

# **COBI REHAB**<sup>®</sup>

The Positioning Company

**COBI REHAB**  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
Tel. +45 7025 2522  
[www.cobirehab.com](http://www.cobirehab.com)