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04/2018/JP

Instructions for using XXL-Rehab Office Chair

CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 To ensure proper use of the XXL-Rehab Office Chair, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .

Also remember that in order to prevent the risk of injury to the user, the chair may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

Safety notes

-  The chair must be used only by the person and for the purpose for which it is intended.
-  The chair is designed for indoor use and should not be exposed to water, other liquids or chemicals.
-  The chair should not be exposed to excessive heat or long and intensive sunlight or other radiation.

Use

XXL-Rehab Office Chair is comfortable and provides good working positions at any place. The seat is well padded, providing good pressure relief and a comfortable sitting position for a long time for bariatric users weighing up to 275 kg.

The user can transfer vertically due to the electric seat hoist and the smooth-running wheels, making it easier to live an active life at home.

 Using the product for individuals weighing more than 275 kg may pose a risk of tipping or damaging the chair.

Description

XXL-Rehab Office Chair is comfortable and provides good working positions at any place.

The user can transfer vertically due to the electric seat hoist and the smooth-running wheels, making it easier to live an active life at home.

Double electric

XXL-Rehab Office Chair has electric height adjustment with double cylinders which makes the hoist extra strong. The hoist has a lift span of either 20 or 28 cm which makes the chair easy to adjust to the correct working height at a kitchen or dinner table. The electric hoist is easy to operate pushing the button located under the armrest.

Strong construction

The chair has a strong safety construction, durable wheels with heavy duty bearings easy to drive and a weight capacity of 275 kg.

The chair has a hand operated brake.

Comfortable seat

The seat is well padded, providing good pressure relief and a comfortable sitting position for a long time. The backrest is reinforced and mounted on a tube which can be adjusted in angle and depth. The armrests can be folded down and have continuously adjustable conical track, ensuring that they never detach.

Cover


The office chair comes with strong upholstery that is stain resistant and fire retardant.

User customisation

The following instructions explain how the XXL-Rehab Office Chair can be customized to the user.


Backrest

The back rest mechanisms have separate controls for height (1) and angle (2). Loosen the levers on both back mechanisms, set the desired height and angle, and then lock the levers again. The medic model can be adjusted in depth with the lever (3). The angle of the spine can be adjusted with the lever (4).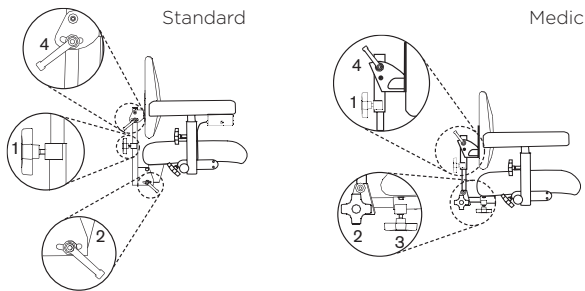

Armrests


The armrests can be adjusted in height and width and tilted backwards. The height is adjusted with button (1). Lower the armrest by turning the button outwardly. Raise it by lifting the armrest to the desired height and then lock button. The width is locked with button (2). Release the button, set it and lock. The tilting is controlled with button (3). Pull it out, and tilt rearwardly. The device automatically locks when the armrest is tilted back to the vertical position. The automatic locking can be prevented by turning (3) button a quarter turn.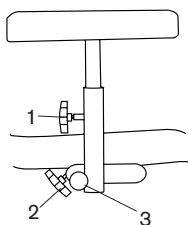

Seat height


The seat height is adjusted with the toggle switch (1) under one armrest.

Raise the seat by pressing the upper part of the power breaker. Decrease by pressing the lower part of the power breaker.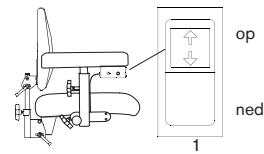

Hand brake

The height is adjusted with the bar (1) at one side of the armrest (standard right). Lift the lever to change the height setting.

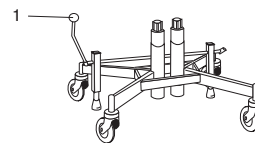Maintenance and cleaning

Regular cleaning of all surfaces of the XXL-Rehab Office Chair will ensure that it continues to look nice and lasts longer.

Of the chair and cover

- Wipe the chair with a dry clean cloth.
- Wash the cover with a furniture detergent.
- For hygienic reasons the seat and padding must also be replaced when the cover is changed.
- Seat, backrest and other upholstered parts are for functional reasons not made of solid material.

Of the electric lifting device

- The electric lifting device must be checked regularly for dirt or dust:
Run up the seat to the highest position. Clean it with a cloth.
- Lubricate then the cylinder with a thin layer of teflon or silicone based lubricant. This type of lubricant does not stain as petroleum-derived fat.

 Do not use water or solvent when cleaning the electric lifting device.

Warranty

The XXL-Rehab Office Chair is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

XXL-Rehab Office Chair is manufactured and tested in accordance with the 93/42/CE directive (class 1) and CEI EN 60601-2-38; UNI EN 1970:2006; 2006/42/CE; CND: z12030799 – RDM: 106290/R – GMDN: 35563.

Specifications

Maximum weight capacity is	275 kg
Frame width	56 cm
Seat width	62 / 72 cm
Seat depth	40 / 45 / 50 cm
Seat height	44-64 / 51-79 cm
Backrest width	46 / 66 cm
Backrest height	43 cm
Adjustable armrest in width	Yes
Adjustable armrest	25-35 cm
Weight	Approx. 40 kg
Cover colour	Grey
Fire retardent cover	Yes
Frame	Powder coated steel
Lifting system	Heavy-duty actuator
Castor	100 mm polyurethane
Cover	100% polyester
Charger	230V
ISO – classification	ISO 18 09 03 - 12

MANUFACTURER

Mercado Medic
Tryffelslingan 14
S-181 57 Lidingö

DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
www.cobirehab.com

04/2018/JP

COBI REHAB[®]

The Positioning Company

COBI REHAB

Fuglebækvej 1D
DK-2770 Kastrup
Denmark

Tel. +45 7025 2522

www.cobirehab.com