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Instructions for XXL-Rehab Patient/Dinner chair

CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 To ensure proper use of the XXL-Rehab Patient/Dinner chair, please read these instructions thoroughly before using the product. Pay particular attention to the warning notes marked with .

Also remember that in order to prevent the risk of injury to the user, the patient/dinner chair should be adjusted only by trained caregivers or aides or by individuals who have been instructed in its proper use.

Use

The XXL-Rehab Patient/Dinner chair is designed to help an individual weighing up to 325 kg when seating is needed e.g. in a hospital, at a dinner table.

 Using the product for individuals weighing more than 325 kg may pose a risk of tipping or damaging the patient/dinner chair.

 Always use the XXL-Rehab Patient/Dinner chair on level, hard flooring with a maximum slope of 3 degrees, as otherwise there is a risk of tipping.

Before use

The bolts for adjusting the legs and backrest must be properly mounted and tightened in the holes on the side of the patient/dinner chair.

 Do not use the patient/dinner chair before the bolts for adjusting the legs and backrest have been properly mounted and tightened, as otherwise there is a risk of the user falling off to the side of the chair.

During use

 To prevent the risk of tipping, only use the patient/dinner chair on a level, hard surface.

 If the user requires assistance during transfers to the patient/dinner chair, the aides must be particularly aware of the risk of performing unsafe lifts or movements when placing the user on the patient/dinner chair. This applies whether the transfer is effected by means of a lift, slide board or support device.

Maintenance and cleaning

The frame components and the seat and armrests may be cleaned with water and a household detergent. To thoroughly clean the seat: remove the seat, clean and make sure to fasten/press down the seat properly again.

Regularly check all bolts and nuts, and retighten if required.

⚠ Failure to inspect the bolts as directed may cause functional failures.

Warranty

The XXL-Rehab Patient/Dinner chair is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

XXL-Rehab Patient/Dinner chair is manufactured and tested in accordance with the international DS/EN 12182:2000 standard.

Specifications

Maximum weight capacity is	325 kg
Total length	80-101 cm
Total outer width on armrests	74 / 84 cm
Total height	79-94 cm
Seat width	61 / 71 cm
Seat depth	40-52 cm
Seat height	44-59 cm
Height of backrest	37 cm
Distance from floor to seat underside	33-48 cm
Distance between armrests	65 / 75 cm
Seat to armrest	28 cm
Weight	16.6 / 17.6 kg
Frame	Powder-lacquered chromium molybdenum steel
Seat / back / armrests	Polyurethane
ISO - classification	ISO 09 33 03

MANUFACTURER AND DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
www.cobirehab.com

04/2018/JP

COBI REHAB[®]

The Positioning Company

COBI REHAB
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel. +45 7025 2522
www.cobirehab.com