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04/2018/JP

Instructions for XXL-Rehab Comfort foam mattress

CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 **To ensure proper use of the XXL-Rehab Comfort foam mattress, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .**

Also remember that in order to prevent the risk of injury to the user, the mattress may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

Use

The XXL-Rehab Comfort foam mattress is designed to provide support and security for a person with a weight of up to 325 kg. The mattress can be used for treatment of pressure relief up to grade 2.

 If users are weighing more than 325 kg there is a risk that the mattress will break or take damage.

Description

The XXL-Rehab Comfort foam mattress is made of two different layers of high quality foam: the base layer is a HD (56Kg/m³) polyurethane foam 15 cm thick. It is manufactured using our patented spring foam technique, based on the distribution of foam springs, which compose the core of the mattress, working independently and distributing the body weight. Recommended by specialists, the system maintains the vertebral spine in the most similar position to an upright position. The system offers a heterogeneous pressure distribution avoiding capillary occlusion. This system avoids mattress deformation and helps to support weights up to 325 kg. The top layer is made of SANITIFOAM[®] viscoelastic foam, it is viscous, very soft to the skin and thermo-sensitive.

This top viscoelastic layer is 3 cm thick and varies the hardness depending on the specific areas that it is placed (pink or white). Pink viscoelastic foam (harder) is placed in the coxal area to distribute the weight in a better way.

Tough and Undistortable: It adapts to the body shape using the body temperature and helping to a uniform pressure distribution.

Anti-allergenic: 100% free of harmful substances. 100% latex free.

Breathable: Open pore manufacturing process allows greater levels of aeration, preventing humidity and the presence of mites and other microbes.

Prevention of pressure ulcers (pu): Body weight distribution minimizes the pressures on the skin allowing unrestricted blood for preventing the appearance of ulcers caused by pressure (pu).

Articulated: This Sanitary Mattress entirely adapts to any kind of Sanitary Bed due to its sectioning design.

Description of cover

The mattress comes covered with an elastic Dhalia fabric cover: 100% polyester coated with PU. The bottom part is made with an antislip beige fabric for preventing the movement of the sanitary mattress on the bed.

In addition, the cover is waterproof, antiallergenic, breathable, antiacaruses and fireproof.

Before use

Place the mattress on the bed. The XXL-Rehab Comfort foam mattress must always be used with the cover zipped correctly.

During use

The mattress can be turned horizontally in the bed to prolong the life of the mattress.

 Inspect the mattress on a regular basis to ensure optimal effect.


Specifications

Maximum weight capacity is	325 kg
Length	200 / 220 cm
Width	100 / 120 cm
Height	18 cm
Weight	12 / 13 kg
Applicable directly on to bed base	Yes
Suitable for adjustable bed base	Yes
Flame-retardant cover	Yes
Incontinence cover	Yes
Antiallergenic cover	Yes
Washable and disinfected cover	Yes
Removable cover	Yes
Mattress	Viscoelastic PUR foam / Heavy Duty PU foam
Cover	100% polyurethane
ISO - classification	ISO 18 12 18 -25

Maintenance and cleaning of cover

- Automatic machine wash to a maximum of 95° C.
- 134° C sterilisation treatment is possible (Autoclaved maximum 3 cycles).
- No bleach, Standard disinfectants may be used.
- Do not iron.
- No dry washing.

⚠ If the cover is damaged do not use the mattress before the cover is replaced.

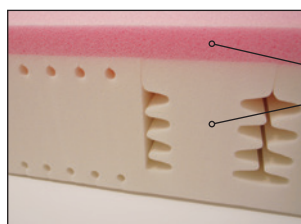Patented design

3 cm viscoelastic foam
15 cm PU foam

Warranty

The XXL-Rehab Comfort foam mattress is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

XXL-Rehab Comfort foam mattress is a SANITARY PRODUCT CLASS I (92/42 ECC), regulated by Royal Decree 414/1996 Part VII Classified as Sanitary Material for the Prevention of Ulcers Caused by Pressure.

The PU and the Viscoelastic foam has the OEKO-TEX Standard 100 CLASS I certification which is a European Standard guaranteeing the omission of harmful substances in the composition of textile products during the entire manufacturing process until final delivery to the client.

MANUFACTURER AND DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
www.cobirehab.com

04/2018/JP

COBI REHAB[®]

The Positioning Company

COBI REHAB
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel. +45 7025 2522
www.cobirehab.com