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04/2018/JP

Instructions for XXL-Rehab Rollator King

CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 To ensure proper use of the XXL-Rehab Rollator King, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .

Also remember that in order to prevent the risk of injury to the user, the rollator may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

Use

1. The rollator can be used for walking support.
2. The rollator can be used as a resting chair when the brakes are locked.

 Using the product for individuals weighing more than 325 kg may pose a risk of tipping or damaging the rollator.

To assemble

Open the box and find the following:

- Walker
- 2 handles only connected to the walker by the cord
- Basket
- Back

1. Unfold the walker.
2. Remove the screws for adjusting the height.
3. Insert the tubes for the handles in the walker.
4. Mount the back onto the walker by pulling the knobs on the left and right side of the walker. By turning the knob 90 degrees you can lock the knob in the "open" position. Slide in the tubes and turn the knob back.

 WARNING: Ensure that the back is securely fastened.

5. The basket is hung onto the upper tube (just below the seat).
6. Finally the height of the handles is adjusted.
7. Ensure that the brakes are correctly adjusted.

To replace or remove backrest

The backrest is placed on the walker to ensure that the weight is not placed too much in front, which can result in tipping of the walker. We strongly recommend to only use the walker with the backrest mounted onto the walker.

1. Pull the knob on the left and right side of the walker. Lock the knob in the "open" position by pulling AND turning the knob 90 degrees.
2. Slide the backrest onto the walker.
3. Make sure to turn back the knobs so they lock the backrest in place.

Reverse procedure if dismantling the backrest.

Adjusting the height of the handles

1. Remove the screws and the knob.
2. Find the required height and insert the screw.
3. Tighten the screw by turning the knob. Make sure the screw fits in the cut-out.

Adjust the height of the handles so that they are even with your hip joint.

⚠ WARNING: Always remember to tighten the hand nuts for height adjustment.

How to fold the walker

1. Grab the front of the seat (the side closest to you when walking) with one hand. Hold one of the handles with the other hand.
2. Lift the seat.
3. The walker now folds.

To minimize the size of an already folded walker, remove the back and basket.

The brakes

The walker can brake in two ways:

1. If you stop for a brief moment: Squeeze the handle and the brakehandle together, so the brake-handle makes an upward move.
2. If you wish to stop for a longer period, e.g. sit and rest, push both brake-handles down, so that they lock in a "down" position. The handles stay in this position until you release them by squeezing the handle and the brake-handle together.

Brake inspection

Your supplier ensures that the brakes are set properly at delivery.

Brake operation, and parking brake should be checked every time before it is used.

If the parking brake cannot lock or if the brakes are not working correctly, you should immediately contact your dealer.

The brakes should have a service check every 3 months.

Walking with the walker

1. Place your hands on the handles. Release the brakes.
2. Then carefully step forward, one foot at a time.
3. Make sure that the surface is even for an easier walk. Pay attention to and be careful about obstacles/holes in the road/surface.
4. Pull the brakes for stopping.

⚠ WARNING: Never use the walker on sloping surfaces that are more than 3,5°.

⚠ WARNING: Never use the walker on soft and uneven surfaces.

Maintenance and cleaning

The frame components, the grips, brakes and the seat may be cleaned with water and a household detergent. Never use organic solvents that could dissolve the grips (such as turpentine, acetone, gasoline, etc).

Regularly check bolts and lock nuts, and retighten if required.

⚠ Failure to inspect the bolts as directed may cause functional failures.

The maximum intended service life: approx. 8 years.

Warranty

The XXL-Rehab Rollator King is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

Warnings

- ⚠ Never use the brakes constantly while walking.
- ⚠ Never sit on the walker and push off with the feet, neither for rolling nor for "fun".
- ⚠ Always lock the walker before sitting down.
- ⚠ Never use the walker on stairs and escalators.
- ⚠ Never use the walker as a wheelchair.
- ⚠ Never use the walker to transport large items - only items that fit in the basket.
- ⚠ Never use the walker on sloping surfaces that are more than 3,5°.
- ⚠ Never use the walker on soft and uneven surfaces .
- ⚠ Always remember to tighten the hand nuts for height adjustment

Specifications

Maximum weight capacity is	325 kg
Total length	85 cm
Total width	71 cm
Width between handles	61 cm
Height adjustable	77.5-99 cm (2.5 cm interval)
Seat width	51 cm
Seat depth	35.5 cm
Seat opening	58 cm
Seat height	50 cm
Weight	15.7 kg
Weight capacity in basket	5 kg
Resting chair	Yes
Foldable	Yes
Massive wheels	Yes
Wheels	8" massive
Frame	Powder-lacquered aluminum
Seat/backrest	Polyurethane
ISO - classification	ISO 12 06 06 - 11

XXL-Rehab Rollator King is manufactured and tested in accordance with the international DS/EN 12182:2000 standard.

MANUFACTURER AND DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
www.cobirehab.com

04/2018/JP

COBI REHAB[®]

The Positioning Company

COBI REHAB

Fuglebækvej 1D

DK-2770 Kastrup

Denmark

Tel. +45 7025 2522

www.cobirehab.com