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04/2018/JP

Instructions for XXL-Rehab Stand Tall

CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 To ensure proper use of the XXL-Rehab Stand Tall, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .

Also remember that in order to prevent the risk of injury to the user, the stand tall may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

Use

The use of the XXL-Rehab Stand Tall is ideal in hospitals or nursing homes.

The XXL-Rehab Stand Tall is designed to provide support and security for persons weighing up to 325 kg when walking indoors.

 Using the stand tall for persons weighing more than 325 kg may increase the risk of tilting or damage the stand tall.

 The XXL-Rehab Stand Tall should be used on a plane hard surface with a maximum inclination of 3.5 degrees, as there may otherwise be a risk of tilting.

Description

The XXL-Rehab Stand Tall is equipped with four horizontal, through-going bolts (two on each side) for height adjustment of forearm supports and hand supports. The bolts are locked with hand nuts.

Final assembly

The height of the stand tall's forearm supports and hand supports is adjustable by screwing the hand nuts off and pulling the two horizontal bolts out of the frame. Then set the forearm and hand supports at the appropriate position, push the bolts through the frame and screw on the hand nuts to secure the bolts.

 The stand tall should not be used until the hand nuts have been screwed tightly onto the bolts, as there may otherwise be a risk of the forearm/hand supports loosening which may cause the user to fall.

Please ensure that the curved heads of the bolts abut closely against the frame, as there may otherwise be a risk that the hand nuts cannot be tightened correctly and therefore work loose.

The hand nuts ought to be facing forward on the stand tall to avoid injury to the user.

During use

Lock the brakes by bringing the locking bracket over the brake handles, while keeping the stand tall braked.

 The brakes must be locked when the user gets up/sits down while using the stand tall as support.

 The brakes may be difficult to activate for users with limited strength of fingers. Such users should be instructed in using the brakes with both hands.

 The stand tall should be manoeuvred carefully across obstacles such as thresholds, as there may otherwise be a risk of tilting.

 The stand tall is not designed for driving up and down ramps inclining more than 3.5 degrees, as there may otherwise be a risk of tilting.

 The stand tall must only be used indoors and on plane hard surfaces to avoid the risk of tilting.

Maintenance and cleaning

The stand tall can be cleaned with water and ordinary cleaning agents.

Lock nuts and bolts must be checked on a regular basis and tightened, if necessary.

⚠ Non-compliance with the bolt inspection guidelines may lead to malfunction.

Specifications

Maximum weight capacity is	325 kg
Total length	73 cm
Total width	76 cm
Width between underarm support, adjustable	68-81 cm
Internal width	72 cm
Adjustable height	100-130 cm
Weight	20 kg
Front wheel	100 mm
Rear wheel	125 mm
Frame	Powder-lacquered chromium molybdenum steel
Grip	PU foam
Underarm support	PU
ISO - classification	ISO 12 06 12 - 06

Warranty

The XXL-Rehab Stand Tall is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

XXL-Rehab Stand Tall is produced and tested in compliance with international standards DS/EN ISO 11199-3:2005 and DS/EN 12182:2000.

MANUFACTURER AND DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
www.cobirehab.com

04/2018/JP

COBI REHAB[®]

The Positioning Company

COBI REHAB
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel. +45 7025 2522
www.cobirehab.com