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04/2018/JP

Instructions for XXL-Rehab Transporter

CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 **To ensure proper use of the XXL-Rehab Transporter, read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .**

Also remember that in order to prevent the risk of injury to the user, the transporter may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

Description

XXL-Rehab Transporter is designed for easy access, independence, safety and on the caregiver terms. The 24" rear wheels provide more directional stability and smooth pushing.

The inside width of the XXL-Rehab Transporter is 71 cm, but using a 5 cm high PU foam seat and 7 cm free space to the armrests allows the body mass to fall down, instead of fading out to the sides. The impossible "wide inside but small outside" is obtained.

The leg supports are designed to carry 100 kg each. The footrests may be adjusted sideways, enabling you to use them with your legs together or apart.

The ability for bariatric users to reach far back to reach the armrest is usually not present. Therefore we have made the armrests with a length of 64 cm and let it stick out 7 cm from the seat base, ready to welcome and support the hands of mobile users. It enables the user to get into the chair by him/herself. The swing away armrests can be taken off.

The space between backrest and seat has a clearance of 32 cm. This gives space for a possible bulbous gluteal region, but also enables the caregivers to position the user better.

Use

The XXL-Rehab Transporter is designed for easy indoor transport inside hospitals and bariatric departments. Another aim with the design was no upkeep and easy to disinfect contra normal wheelchairs.

The basket under the chair is made so that belongings etc can follow the patient during transit.

 Using the product for individuals weighing more than 325 kg may pose a risk of tipping or damaging the transporter.

 Always use the XXL-Rehab Transporter on level, hard flooring with a maximum slope of 3 degrees, as otherwise there is a risk of tipping.

Before use

The armrests must be properly installed in the sockets on the side of the transporter.

The width of the footrests may be gradually adjusted. The lock bolts beneath the foot plates must always be properly mounted in the footrest sockets and must not be secured to the footrests outside the sockets.

⚠ To prevent the risk of the user falling off the side of the transporter, never use it without first properly installing the armrests.

During use

⚠ Never stand on the foot plates or use them as a platform during transfers, as this will pose a high risk of tipping.

Pinch hazards

- Installing the armrests while the user is seated on the transporter may present a pinch hazard between the armrests and the side of the transporter.
- There is a risk of pinching a foot between the foot plates if the lock bolts are not properly mounted in the footrest sockets.

⚠ To prevent the risk of tipping, only drive the XXL-Rehab Transporter on a level, hard surface.

⚠ Both wheels must be locked with the wheel locks when the user is seated on the transporter, except when it is used for transport.

⚠ If the user requires assistance during transfers to the XXL-Rehab Transporter, the aides must be particularly aware of the risk of performing unsafe lifts or movements when placing the user in the chair. This applies whether the transfer is effected by means of a lift, slide board or support device.

Maintenance and cleaning

The white frame components and the seat and armrests may be cleaned with water and a household detergent.

To thoroughly clean the seat: unscrew the four screws under the seat, clean and make sure to fasten the seat properly again.

Regularly check bolts and lock nuts, and retighten if required.

 Failure to inspect the bolts as directed may cause functional failures.

Warranty

The XXL-Rehab Transporter is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

XXL-Rehab Transporter is manufactured and tested in accordance with the international DS/EN 12182:2000 standard.

Specifications

Maximum weight capacity is	325 kg
Seat width with standard armrests	71 cm
Total outer width with standard armrests	78 cm
Seat depth	56 cm
Seat height	55 cm
Backrest	32-52 cm
Seat to footrest	40-42-44-46-48 cm
Width of footrest (total width changes)	58-110 cm
Pushbar from floor	100 cm
Width between armrests	54 cm
Height of backrest	48 cm
Height of footrests	40-48 cm
Distance from floor to seat underside	44.5 cm
Total length	129 cm
Total weight	30 kg
Front wheels	125 mm
Rear wheels	24"
Seat	Pressure moulded PUR foam
Frame	Powder-lacquered chromium molybdenum steel
Footrest cover	PU foam
ISO - classification	ISO 09 12 03

MANUFACTURER AND DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
www.cobirehab.com

04/2018/JP

COBI REHAB®

The Positioning Company

COBI REHAB

Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel. +45 7025 2522
www.cobirehab.com