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04/2018/JP

Instructions for using XXL-Rehab 500 bariatric hospital bed

CAUTION

This product should not be used without proper instruction from a healthcare professional.

Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.

 To ensure proper use of the XXL-Rehab 500 bariatric hospital bed, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .

Also remember that in order to prevent the risk of injury to the user, the bed may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.


Intended use

The XXL-Rehab 500 bariatric hospital bed can be used for bariatric patients with a maximum weight of up to 450 kg.


The bed is particularly suitable for hospitals, home care and rehabilitation centers.

Specifications

Maximum weight-bearing capacity is	450 kg + 50 kg equipment
Total length	231-251 cm
Total width	100-128 cm
Mattress size length	200 / 220 cm
Mattress size width	90 / 100 / 120 cm
Manually adjustable in length	200 / 220 cm
Manually adjustable in width	90 / 100 / 120 cm
Lying surface height	39-82 cm
Minimum clear distance from the floor	15 cm
Total weight	175 kg
Height adjustable	Yes
Separate control for caregiver e.g. Trendelenburg and Anti-Trendelenburg position	Yes
Electrical adjustment of backrest, leg section, thigh section, height of lying surface	Yes
Headboard up / down feature	Yes
Foot end up / down feature	Yes
Side rails removable without tools	Yes
Direction control on the wheels	Yes
Central brake	Yes
Magnetic lock on the user hand control	Yes
Built-in bed extension	Yes
Lock of individual electric functions performed from the caregiver hand control	Yes
Separate adjustment of backrest	Yes
Coupled adjustment of lying surface	Yes
Removable lifting pole	Yes
Removable foot gable	Yes
High slatsides	Yes
Frame	Powder-coated steel
Wheel diameter	145 mm
Ingress for water	66 IP
ISO - classification	ISO 18 12 10 - 30

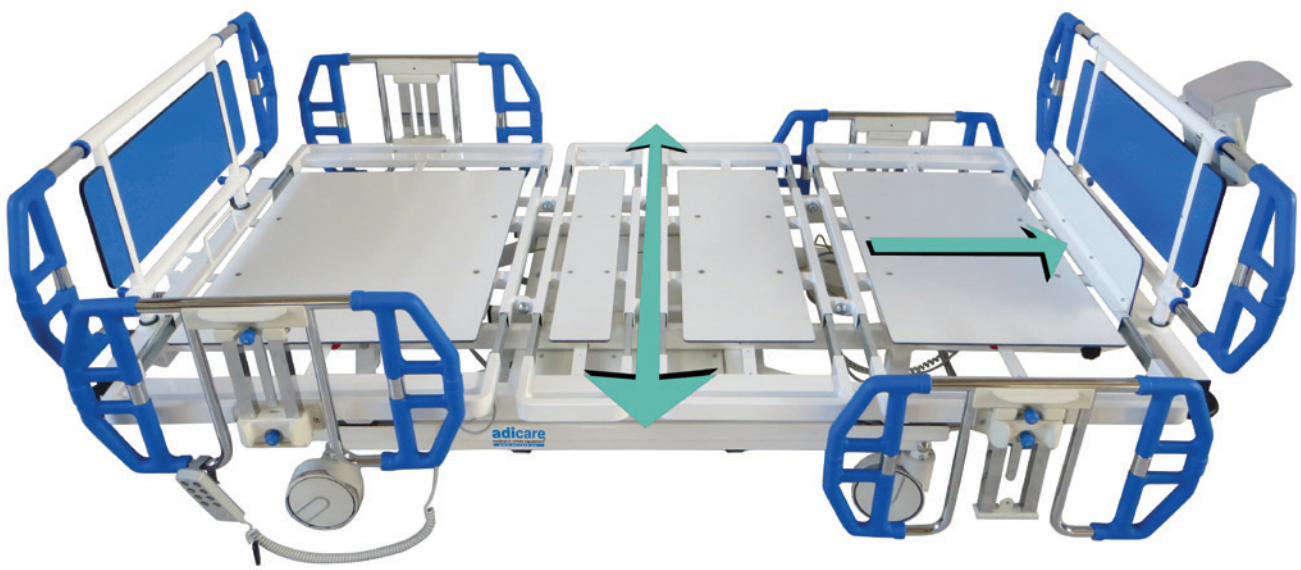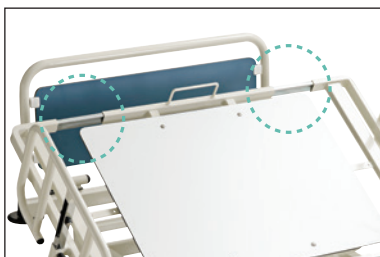The bed can have a maximum load of 500 kg, including user and equipment.

Adjustable in width at both sides.

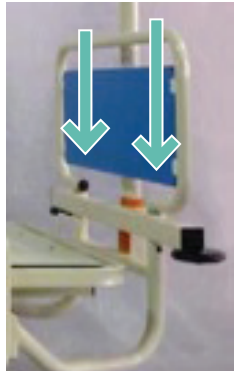
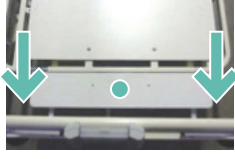
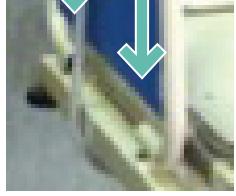
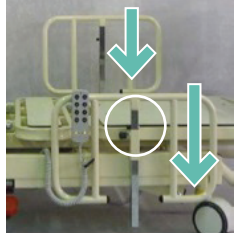
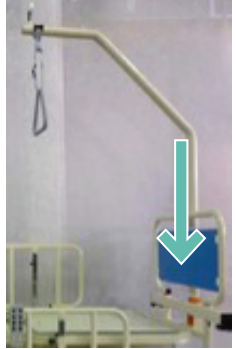
Caregiver control panel.

User hand control.

Component list to assemble

Assembling of head board Insert head board in the proper sockets on bed main frame, as red arrows show in the picture. TECHNICAL NOTES Head board sockets are designed to guarantee fast and reliable emergency removal.		Section widening – in width The bed can be adjusted in width from 90/100/110/120 cm. Sections can be adjusted to 2 different sizes by a comfortable and reliable “clickclack” device as shown in the picture: just pull the black knob with a hand and with the other hand pull out the section to the desired size, release the black knob and verify if the section is locked. Remember to adjust each section.		
Assembling of foot board Insert foot board in the proper sockets on bed main frame, as red arrows show in the picture. TECHNICAL NOTES Head board sockets are designed to guarantee fast and reliable emergency removal.		Bed extension – in lenght Turn the two black hand wheel knobs (right and left side) below the foot section and pull the foot section out, then extract the mattress holder from its socket and place it as shown in the picture.		
Caregiver control panel 1 Attach the caregiver control panel to the upper part of the foot board. The caregiver control panel can quickly be removed in case of an emergency situation.		Patient control panel Use of patient control panel can be disabled with the included magnetic key (M) by pulling it over “lock” symbol on the control panel (SZ). Lamp lights up: control panel is activated. Lamp off: control panel is disabled / locked.		
Adjustment of side rails Insert each side rail in its proper socket, as red arrows show in the picture. They can be adjusted at 3 different heights by a comfortable and reliable “click-clack” device as red circle shows in the picture: just pull the black knob, lift up the side rail, take it to the desired height, release the black knob and verify if the side rail is locked.				
Assembling of lifting pole with IV hooks Insert bariatric lifting pole with IV hooks in its proper socket behind head board, on main frame as red arrow shows in the picture. TECHNICAL NOTES Bariatric lifting pole with IV hooks is made of 2 high-strength steel tubes.				

Component list to assemble

Caregiver control panel

S N T

On the caregiver control panel there are 3 locking buttons above the electronic functions. By turning the locking knob in horizontal position the function of both patient and caregiver control panel is blocked.

"S" button is activated by the caregiver to prevent that the electronic functions are used simultaneously.

Red key "N": Emergency; brings the bed down.

Red Key "T": Trendelenburg position; brings the bed in the above position.

Back rest (0°-54°)

1 2

Push button 1 with the equivalent explanatory graphic to RAISE the BACKREST as shown in the picture (HANDSET). Push button 2 to LOWER the BACKREST.

Leg rest (0°-24°)

3 4

Push button 3 with the equivalent explanatory graphic to RAISE the LEGREST as shown in the picture (HANDSET). Push button 4 to LOWER the LEGREST.

Back rest & leg rest simultaneously

5 6

On Patient control panel push button 5 with the equivalent explanatory graphic to RAISE BACKREST & LEGREST SIMULTANEOUSLY as shown in the picture (HANDSET). On the patient control panel push button 6 to LOWER BACKREST & LEGREST SIMULTANEOUSLY.

Adjusting height (37-80 cm)

7 8

Push button 7 with the equivalent explanatory graphic to RAISE THE BED as shown in the picture (HANDSET). Push button 8 to LOWER THE BED.

Trendelenburg position (max +/- 18°)

9

Push button 9 with the equivalent explanatory graphic to get TRENDLENBURG position as shown in the picture (SUPERVISOR).

Continuous beep sound

The operation is blocked or the function is completed.

Wheel steering

A 3 step adjustable central braking pedal is placed on the steering wheels in the foot end of the bed.

Steering position

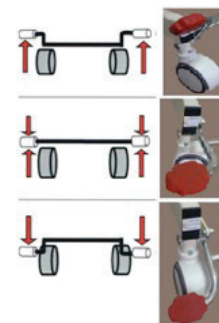
Push the pedal up until the steering castor is engaged. This adjustment allows driving the bed on a straight line.

Swivelling position

Push the pedal in the neutral position, castors are free to swivel, they are all unlocked.

Braked position

Push the pedal down, castors are all locked now.

Maintenance (3 times a year) and cleaning

- After a week or two of usage, check all the bolts to be sure that none of them have worked loose and that all pins are in their normal location and securely fastened.
- Check all welds.
- Castors with brake: often check the brake blocks.
- Often check that screws and nuts are kept well tightened down.
- Use mild soap and water or a disinfectant to clean the bed. Wet a cloth with the cleaning product and wring dry.

 **Failure to inspect the bolts as directed may cause functional failures.**

Accessories

Side rails, lifting pole and IV pole.

Warnings

Inform the patient of these rules of security:

-  The bed is only suitable for people over 12 years old.
-  The safe working load (mattress included) is 450 kg user weight + 50 kg equipment.
-  Persons who have not read the user manual must not use the bed.
-  It is not allowed to use the bed for other purposes than the ones stated in INTENDED USE.
-  For your safety, during bed transfers, be sure that the sections are in flat position and the height of the bed is at least 55 cm from the ground.
-  Remove any objects or obstacles that might be present in the working area around the bed.
-  Remember to lock the castors after moving the bed.
-  When not in use hook the hand control on the top of the side rails.
-  Make sure that all nuts, bolts and screws are kept well tightened.
-  For safety reasons, make sure to replace worn-out or damaged parts in due time.
-  Failure to inspect the bolts as directed may cause functional failures.

Warranty

The XXL-Rehab 500 bariatric hospital bed is covered by one year warranty from the invoice date for materials and workmanship.

Cobi Rehab guarantees that the product is flawless when it is delivered. If the product has a defect or is damaged when it is delivered, you should immediately contact Cobi Rehab.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void. Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

This warranty does NOT cover normal maintenance such as cleaning, adjustment, lubrication and updating of equipment or parts.

XXL-Rehab 500 bariatric hospital bed is CE marked in accordance with 93/42/CE directive (class 1) and CEI EN 60601-2-38; UNI EN 1970:2006; 2006/42/CE; CND: z12030799 – RDM: 106290/R – GMDN: 35563.

MANUFACTURER

Ascuro AG
Seewenweg 6
CH-4153 Reinach

DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
www.cobirehab.com

04/2018/JP

COBI REHAB[®]

The Positioning Company

COBI REHAB

Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel. +45 7025 2522
www.cobirehab.com