

XXL-Rehab Bariatric rehabilitation Shower and toilet

*Positioning solutions for bariatric patients
all hours of the day*



COBI REHAB®
The Positioning Company



COBI REHAB®
The Positioning Company

XXL-Rehab Shower commode

We have aimed at designing the best shower commode on the market. It is designed for easy access, independence, safety and on the caregiver terms.

The inside width of the chair is 61 or 71 cm, but using a 5 cm high PU foam seat and 7 cm free space to the armrests allows the body mass to fall down, instead of fading out to the sides. The impossible “wide inside but small outside” is obtained. This allows the XXL-Rehab Shower Commode to pass through nearly all modern doorways.

The footrest is always the weak point when it comes to bariatric products. Normally the producer makes a short cut here, by using a standard solution. We did not only design the footrest to carry 100 kg each, we did more!

The footrest is also side/width adjustable, so access to intimate hygiene is possible and people with the fat-tissue placed on the inside of their leg can sit with legs apart (up to 110 cm) and still have foot support. The brand new footrest is even nicely covered with PU foam.

One of the unique new features is a split toilet seat, a long open hole: 44 cm and 22 cm at the widest point. This to enable a correct use of the chair, it eases access for caregivers and avoids incidents where material ends up in places it wasn't meant to end up!

The ability for bariatric users to reach far back to reach the armrest is usually not present. Therefore we have made the armrests with a length of 64 cm and let it stick out 7 cm from the seat base, ready to welcome and support the hands of mobile users. It enables the user to get into the chair by him/herself. Another option is the butterfly armrests. Butterfly armrests add 14 cm to the seating width.

The space between backrest and seat has a clearance of 32 cm. This gives space for a possible bulbous gluteal region, but also enables the caregivers to reach for the sling and position the user better.

- Long armrests provide easy access
- Extra seat width with butterfly armrests

MAX 325 kg

Product video
Scan the code and see
the product in use



XXL-Rehab Shower commode

Specifications

XXL-Rehab Shower commode	
Maximum weight capacity is	325 kg
Seat width with standard armrests	61 / 71 cm
Total outer width with standard armrests	68 / 78 cm
Seat depth	56 cm
Seat height	55 cm
Backrest	32-52 cm
Seat to footrest	40-42-44-46-48 cm
Width of footrest	58-110 cm
Pushbar from floor	100 cm
Width between armrests	54 cm
Height of backrest	48 cm
Height of footrests	40-48 cm
Distance from floor to seat underside	44.5 cm
Total length	117 cm
Total weight	27 / 30 kg
Wheels (2 wheels with individual locks)	125 mm
Seat	Pressure moulded PUR foam
Frame	Powder-lacquered chromium molybdenum steel
ISO - classification	ISO 09 12 03

Ideal use for...

- Bariatric users.
- Bariatric departments.
- Institutions.
- Home.
- Hospitals.



Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Shower commode incl. standard armrests, standard footrests and commode bucket	61	56	0150-061-000
XXL-Rehab Shower commode incl. standard armrests, standard footrests and commode bucket	71	56	0150-071-000
XXL-Rehab Shower commode incl. butterfly armrests, standard footrests and commode bucket	61 + 14	56	0151-061-000
XXL-Rehab Shower commode incl. butterfly armrests, standard footrests and commode bucket	71 + 14	56	0151-071-000



Seat with lid



Split toilet seat

Accessories	
Butterfly armrest left (add 14 cm to seat width)	0150-999-012L
Butterfly armrest right (add 14 cm to seat width)	0150-999-012R
Commode bucket	0140-099-001
Standard footrest right	0156-999-090H
Standard footrest left	0156-999-090V
Calf support / amputation support right	0156-999-030
Calf support / amputation support left	0156-999-031
Full size seat for shower commode with lid for bucket	0150-999-023



XXL-Rehab Shower commode wheeler

We have aimed at designing the best shower commode wheeler on the market. It is designed for easy access, independence, safety and on the caregiver terms. The 24" rear wheels provide more directional stability and smooth pushing.

The inside width of the chair is 61 or 71 cm, but using a 5 cm high PU foam seat and 7 cm free space to the armrests allows the body mass to fall down, instead of fading out to the sides. The impossible "wide inside but small outside" is obtained.

The footrest is always the weak point when it comes to bariatric products. Normally the producer makes a short cut here, by using a standard solution. We did not only design the footrest to carry 100 kg each, we did more!

The footrest is also side/width adjustable, so access to intimate hygiene is possible and people with the fat-tissue placed on the inside of their leg can sit with legs apart (up to 110 cm) and still have foot support. The brand new footrest is even nicely covered with PU foam.

One of the unique new features is a split toilet seat, a long open hole: 44 cm and 22 cm at the widest point. This to enable a correct use of the chair, it eases access for caregivers and avoids incidents where material ends up in places it wasn't meant to end up!

The ability for bariatric users to reach far back to reach the armrest is usually not present. Therefore we have made the armrests with a length of 64 cm and let it stick out 7 cm from the seat base, ready to welcome and support the hands of mobile users. It enables the user to get into the chair by him/herself. Another option is the butterfly armrests. Butterfly armrests add 14 cm to the seating width.

The space between backrest and seat has a clearance of 32 cm. This gives space for a possible bulbous gluteal region, but also enables the caregivers to reach for the sling and position the user better.



- XXL-Rehab Shower commode wheeler with 24" rear wheels is easy to drive for the attendant

MAX 325 kg

XXL-Rehab Shower commode wheeler

Specifications

XXL-Rehab Shower commode wheeler	
Maximum weight capacity is	325 kg
Seat width with standard armrests	61 / 71 cm
Total outer width with standard armrests	68 / 78 cm
Seat depth	56 cm
Seat height	55 cm
Backrest	32-52 cm
Seat to footrest	40-42-44-46-48 cm
Width of footrest	58-110 cm
Pushbar from floor	100 cm
Width between armrests	54 cm
Height of backrest	48 cm
Height of footrests	40-48 cm
Distance from floor to seat underside	44.5 cm
Total length	129 cm
Total weight	27 / 30 kg
Front wheels	125 mm
Rear wheels	24"
Seat	Pressure moulded PUR foam
Frame	Powder-lacquered chromium molybdenum steel
ISO - classification	ISO 09 12 03

Ideal use for...

- Bariatric users.
- Bariatric departments.
- Institutions.
- Home.
- Hospitals.




Wheel lock



Calf support /
amputation support

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Shower commode wheeler incl. standard armrests, standard footrests and commode bucket	61	56	0152-061-000
XXL-Rehab Shower commode wheeler incl. standard armrests, standard footrests and commode bucket	71	56	0152-071-000
XXL-Rehab Shower commode wheeler incl. butterfly armrests, standard footrests and commode bucket	61 + 14	56	0153-061-000
XXL-Rehab Shower commode wheeler incl. butterfly armrests, standard footrests and commode bucket	71 + 14	56	0153-061-000

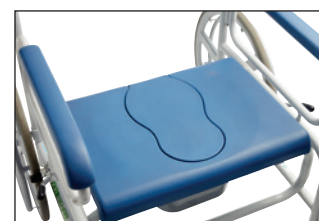

Seat with lid



Split toilet seat

Accessories	
Butterfly armrests left (add 14 cm to seat width)	0150-999-012L
Butterfly armrest right (add 14 cm to seat width)	0150-999-012R
Commode bucket	0140-999-001
Standard footrest right	0156-999-090H
Standard footrest left	0156-999-090V
Full size seat for shower commode with lid for bucket	0150-999-023
Calf support / amputation support right	0156-999-030
Calf support / amputation support left	0156-999-031



COBI REHAB®
The Positioning Company

XXL-Rehab Shower Commode Rise N Tilt

XXL-Rehab Shower Commode Rise N Tilt is a combination chair with built-in lift and tilt function. It is the world's first shower commode for bariatric users, which can both lift and tilt the user to a 30° degree. The shower commode has a weight capacity of 325 kg.

Making tasks easier and safer

XXL-Rehab Shower Commode Rise N Tilt provides better access for caregivers in relation to hygiene, skin and tissue care. Nursing and dressing of feet may, for caregivers, take place in a standing position, thus achieving an ergonomically sound working position. The shower commode also allows the caregiver and the user to have eye contact, which provides far better communication.

Improved and more efficient care

This shower commode raises the efficiency of care because it enables a single caregiver to perform the whole showering procedure, as well as a wide range of everyday hygiene tasks.

The high level of support provided by the shower commode results in streamlined routines that offer considerable time savings compared with traditional methods.

XXL-Rehab Shower Commode Rise N Tilt is designed to give bariatric users a comfortable and safe bathing experience and can also be used where the user's condition is poor with reduced postural control.

Enhancing quality of life

XXL-Rehab Shower Commode Rise N Tilt can enhance the quality of life for a bariatric user by making assisted showering and other hygiene routines not only safer and more comfortable, but also a more dignified experience.

- Lift and tilt function up to 30°
- Possibility of extra wide butterfly armrests
- Brakes on all 4 wheels
- Unique lift and tilt function
- Hand control gives the caretaker full control
- High backrest and sturdy armrests

MAX 325 kg



In lowest position with standard footrests

XXL-Rehab Shower Commode Rise N Tilt Specifications

XXL-Rehab Shower Commode Rise N Tilt	
Maximum weight capacity	325 kg
Seat depth adjustable	42 / 48 / 54 cm
Total length (lowest position)	140 cm
Total width	74 cm
Seat width	66 cm
Seat height	51-69 cm
Electric tilt	0-30°
Tilt of seat independent from height adjustment	Yes
Width between standard armrests	61 cm
Width between butterfly armrests	76 cm
Width toilet opening	Diameter 24 cm, 35 cm long
Seat height electric adjustable	55-100 cm
Distance from floor to seat underside	44.5 cm
Pushbar from floor	100 cm
Seat to footrest	39-42-45-48 cm
Height seat to armrest	22 cm
Weight	46 kg
Swing-away armrests	Yes
Swing-away footrests	Yes
Removable footrests	Yes
Height adjustable footrests	Yes
Headrest	Yes
4 wheels with individual locks	Yes
Control	Via the handset
Capacity up/down fully charged battery	Approx. 15 times
Turning radius	156 cm
Frame	Powder lacquered chromium molybdenum steel
Wheels	Diameter 150 mm
Seat / back / armrests	Polyurethane
IP - classification	IPX6
IPX6	Washable, also in washing tunnel, 20°C IEC 60601-2-52
ISO - classification	ISO 09 12 03 - 23

Ideal use for...

- Bariatric users.
- Users with reduced muscle tone.
- Pressure sores.
- Spasm / involuntary movements.
- Amputations.
- Stroke.



Commode bucket with lid



Seat with lid



Split toilet seat

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Shower Commode Rise N Tilt incl. standard footrests, charger, headrest and commode bucket	66	42 / 48 / 54	0157-066-000
Accessories			
Butterfly armrest right (add 14 cm to seat width)			0150-999-012R
Butterfly armrest left (add 14 cm to seat width)			0150-999-012L
Headrest adjustable			0156-999-020
Full size seat for shower commode with lid for bucket			0150-999-023
Commode bucket			0140-999-001
Standard footrest right			0156-999-090H
Standard footrest left			0156-999-090V
Calf support / amputation support right			0156-999-030
Calf support / amputation support left			0156-999-031
Battery for XXL-Rehab Rise N Tilt			0157-999-050

The handset gives the caregiver control over battery-powered functions





XXL-Rehab Shower Commode Tilt

The XXL-Rehab Shower Commode Electric Tilt has a 32 degree tilt position and a weight capacity of 325 kg Safe Working Load. This shower commode is designed to provide a comfortable alternative showering system in instances where the prone position might cause discomfort or distress to the user. It may also be used in circumstances of limited space providing a bathing solution where other units may not have room for operation.

Making tasks easier and safer

The XXL-Rehab Shower Commode Tilt provides better access for the caregiver in terms of hygiene, skin and tissue care as well as nursing/dressing feet in standing position, allowing the caregiver to work in a safe and ergonomically sound way.

Improving care, efficiency and communication

This shower commode raises the efficiency of care because it enables a single caregiver to perform the whole showering procedure, as well as a wide range of everyday hygiene tasks. The high level of support provided by the shower commode results in streamlined routines that offer considerable time savings compared with traditional methods. It also enables the caregiver and the user to maintain eye contact for better communication.

Suitable for

The XXL-Rehab Shower Commode Tilt is suitable for patients with reduced muscle tone, pressure ulcers, spasm, involuntary movement, amputations or stroke patients.

- Electric tilt up to 32°
- The high backrest and sturdy armrests provide optimum user support and security
- The reclined position gives full access for the caregiver to most parts of the body and is very comfortable for the user

MAX 325 kg

Product video
Scan the code and see
the product in use



With standard footrests

XXL-Rehab Shower Commode Tilt Specifications

XXL-Rehab Shower Commode Tilt	
Maximum weight capacity is	325 kg
Seat depth adjustable	42 / 48 / 54 cm
Total length (lowest position)	140 cm
Total width	74 cm
Seat width	66 cm
Seat height	51-69 cm
Backrest	24-53 cm
Width of footrest	58-110 cm
Height of footrests	40-48 cm
Electric tilt	0-32°
Width between standard armrests	61 cm
Width between butterfly armrests	76 cm
Width toilet opening	Diameter 24 cm, 35 cm long
Distance from floor to seat underside	44.5 cm
Pushbar from floor	100 cm
Seat to footrest	39-42-45-48 cm
Height seat to armrest	22 cm
Weight	39 kg
Swing-away armrests	Yes
Swing-away footrests	Yes
Removable footrests	Yes
Height adjustable footrests	Yes
Headrest	Yes
4 wheels with individual locks	Yes
Control	Via the control panel
Capacity up/down fully charged battery	Approx. 20 times
Turning radius	158 cm
Frame	Powder lacquered chromium molybdenum steel
Wheels diameter	150 mm
Seat / back / armrests	Polyurethane
IP - classification	IP44
ISO - classification	ISO 09 12 03 - 23

Ideal use for...

- Bariatric users.
- Users with reduced muscle tone.
- Pressure sores.
- Spasm / involuntary movements.
- Amputations.
- Stroke.



Headrest



Seat with lid



Split toilet seat

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Shower Commode Tilt incl. standard footrests, charger, headrest and commode bucket	66	42 / 48 / 54	0155-066-000
Accessories			
Butterfly armrest left (add 14 cm to seat width)			0150-999-012L
Butterfly armrest right (add 14 cm to seat width)			0150-999-012R
Headrest adjustable			0156-999-020
Full size seat for shower commode with lid for bucket			0150-999-023
Commode bucket			0140-999-001
Standard footrest right			0156-999-090H
Standard footrest left			0156-999-090V
Calf support / amputation support right			0156-999-030
Calf support / amputation support left			0156-999-031



The handset gives the caregiver control over battery-powered functions



COBI REHAB®
The Positioning Company

XXL-Rehab Clean

Strongly engineered and functionally designed exclusively for bariatric users!

A combination chair with toilet opening. When the seat cover is removed, the chair becomes a bedside commode

We have designed the armrests to bend out like a butterfly, so the body doesn't get squeezed, getting in and out the chair. The butterfly armrests of the XXL-Rehab Clean also provide much better functionality for the user, compared to other chairs with straight and short armrests where it is almost impossible for a bariatric user to get a good grip.

The height of the chair is adjustable from 44 to 59 cm. It is very important to adjust the chair to the right height to have success getting in and out of the chair. If the chair is too high for the user, the feet will lift off the ground, making it hard to get seated in the chair. If the chair is too low, the user will have trouble getting up.

It is often a problem for bariatric users to be seated comfortably. Sometimes the seat depth is too short, sometimes too deep and with no back support. We have designed the seat to be individually adjusted to provide a comfortable seating. The backrest can be moved +/- 6 cm and the bended tubing assures that the right comfort can be found.

It is easy to clean the XXL-Rehab Clean with its removable seat pad. No tools are needed and the bedpan is redesigned for a smoothly hygienic cleaning operation.

It is also made easy to both handle and move the chair from place to place. In the construction of the chair we have only used high-tech lightweight tubing, to provide the highest strength at the lowest weight.

The commode bucket is included when purchasing the chair.



Seat with lid



Split toilet seat

- Adjustable seat height
- Adjustable seat depth
- Long stable armrests
- 2 seat widths
- Combined bedside commode and shower bench

MAX 325 kg

Ideal use for...

- Bariatric users.

XXL-Rehab Clean	
Maximum weight capacity is	325 kg
Seat width	61 / 71 cm
Total length	80-101 cm
Seat depth	40-52 cm
Seat height	44-59 cm
Backrest height	37 cm
Distance from floor to seat underside	33-48 cm
Total outer width of front legs	69 / 79 cm
Total height	79-94 cm
Total weight	15 / 16 kg
Total outer width on armrests	73 / 83 cm
Seat / back / armrests	Polyurethan
Frame	Powder-lacquered chromium molybdenum steel
ISO - classification	ISO 09 33 03

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Clean	61	40-52	0142-061-000
XXL-Rehab Clean	71	40-52	0142-071-000
Accessories			
Commode bucket			0140-999-001

Content

2-3	XXL-Rehab Shower commode
4-5	XXL-Rehab Shower commode wheeler
6-7	XXL-Rehab Shower Commode Rise N Tilt
8-9	XXL-Rehab Shower Commode Tilt
10	XXL-Rehab Clean



COBI REHAB[®]
The Positioning Company

COBI REHAB
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel. +45 7025 2522
www.cobirehab.com