

LASAL

May 2018



LASAL - the solution to positioning

POSITIONING



LASAL A/S, Industrivej 15, DK-8881 Thorsoe

E-mail: info@lasal.dk



+45 8696 6666



www.lasal.dk

VAT No. DK 1437 8677

LASAL Cushions is the right choice when it comes to positioning.

Working with persons with brain damages, neurological patients or persons with reduced mobility LASAL cushions give the caretaker a great tool to create both support and comfort in almost any situation.

The correct posture can prevent formation of contractures, reduce spasticity and prevent pressure sores.

LASAL Positioning Cushions are extremely suitable in these areas as they have a unique ability to adjust to the body shape as well as to the supporting surface.



Goals of positioning

- **Correcting and stabilizing the body segments in relation to each other and the supporting surface.**
- **Controlling the forces that affect the body allowing a functional position and increased comfort.**

The cushions provide support in various positionings, but are also very ideal for delimitation and increased body-consciousness. With LASAL Positioning Cushions the user will experience comfort, safety, pleasant support of the body without being pinned down.

For the helper a LASAL Positioning Cushion is an easy-to-use tool facilitating more varied positioning and the possibility of changing posture more often.

The cushions are placed tightly around the person and provides a feeling of delimitation of the body, which is calming.

LASAL Positioning Cushions:

- Are mouldable and adapt to the posture
- Support and stabilize
- Are available in multiple shapes and sizes that give numerous variation possibilities
- Reflect body heat and stimulate the muscles and circulation
- Are easily adjusted to the every single positioning
- Come with outer cover made from 100% ÖKO TEX certified cotton
- Divided into colour codes as an aid to positioning
- Are easy to clean

T-Cushion



The T-Cushion is used as an abduction cushion, between the legs, or for support in general of the lower limbs. Can also be used for prone position, where it lifts the person up into a convenient height from the supporting surface making the position more comfortable and giving the shoulders more freedom of movement.

Banana-shaped Cushions



The Banana-shaped Cushion is a multipurpose cushion with a myriad of applications in positioning. Whether you want a sitting or lying position and depending on the desired stabilization Banana Cushion can be used alone or combined with other LASAL Positioning Cushions. Banana Cushion is moderately filled, which makes it mouldable. The filling is distributed so that it fits into the desired position. Banana Cushions can be used by both children and adults for positioning in the resting positions or e.g. during training and therapeutic exercise. The cushions are also suitable to encircle the user helping to create calmness and security.

Triangular Cushions



The Triangular Cushion is a relatively firm cushion and can be used as a stabilizer along the body in a seated or lying position or as a filling in the bed.

The smaller Triangle Cushions can be used for other purposes such as supporting under the knees or along the lower limbs.

Square Cushions



The Square Cushions are mouldable universal cushions particularly suited to support the head, arms, legs and feet. For positioning of stroke patients the Square Cushion is suitable to relieve the weak side while maintaining a comfortable position.

Square Cushions are available in more sizes and can be used by both children and adults.

Half Moon Cushions



Half Moon Cushions are used for support of the arm and hand, in the arch, in both sitting and lying position and for pressure relief between the knees and feet.

Neck Collar



Neck Collar is placed close up against the neck in a ring. It supports and stabilizes the head and neck region in a partly sitting position. The collar is mouldable and the filling can be pushed to where it is needed the most.

Cover can be ordered with grip tape closure, whereby the collar can be closed more closely to the neck.

Neck Cushions



The more U-shaped Neck Cushion lie loosely on the shoulder and supports neck and head in lying and supported sitting position. The Neck Cushion can also be used to stabilize a hemiplegic arm when sitting or lying on the side.

900 can advantageously be used for smaller children to encircle the body and stabilize a desired position. Placed under the knees and up along the hip the cushion can help to stabilize and correct the hip and pelvic region.

Support Cushions



Support Cushions are available in three sizes and are slightly chubby and oblong shaped and wedge shaped in height. With these cushions come multiple applications for both children and adults. They can all be used as seat cushions, where the special wedge shape and the recess in the center affects the pelvis rotation and stabilizes the seat position.

The size of 400 and 420 makes them useful as small sit sacks for children. Furthermore the seat cushion placed on a skateboard or on the floor and thus constitute an active equipment for exercise or play.

Position for hemi paretic



Support for hip / knee / feet



Abduction



30 degree on the side front



30 degree on the side back



Support by sitting



Prone



Supine



In a wheel chair



On the side



Girl with Lax Spastic in prone position with T- and Moon Cushion



Recovery position using a Banana Cushion and Neck Collar for head support




Note the head position



Girl with proper head position using a small Half Moon and Cushion No. 900 as support for arms



Boy with severe Scoliosis and contractions

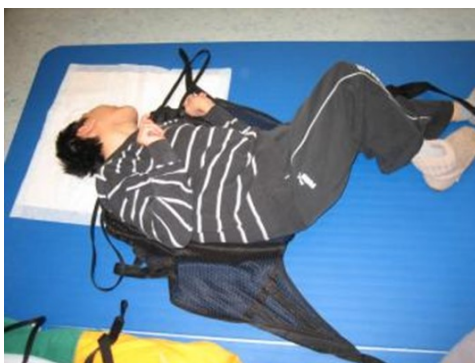

Positioned on the back with Bananas and T-Cushion



Girl with severe Scoliosis and Hip Lux



30 degree positioning with Banana, U-cushion and small Moon




Cushions for active use: Long Sit Cushion and T-Cushion



Specifications


T-Cushions

Item No.	Name of Product	Colour of outer cover	Length A/B	Width A/B	Height
6000	T-Cushion, XL	Blue	98 / 26 cm	61 / 29 cm	11 cm
6010	T-Cushion, L	Marine blue	83 / 27 cm	56 / 27 cm	10 cm
6015	T-Cushion, L/M	Turquoise	72 / 25 cm	45 / 20 cm	8 cm
6020	T-Cushion, M	Green	63 / 22 cm	40 / 19,5 cm	7 cm
6030	T-Cushion S	Red	56,5 / 17,5 cm	37 / 15 cm	7 cm
6035	T-Cushion, S/XS	Blue	46,5 / 16 cm	32 / 15 cm	5 cm
6040	T-Cushion, XS	yellow	36 / 13 cm	27 / 12,5 cm	4 cm

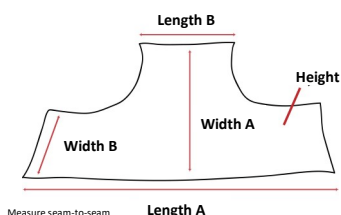

Banana Cushions


Item No.	Name of Product	Colour of outer cover	Length	Inner Length	Width	Height
100	Banana Cushion, L	Yellow	-	148 cm	31 cm	10 cm
110	Banana Cushion, S	Red	-	144 cm	40 cm	12 cm
150	Snake-shaped Cushion	Green	320 cm	-	30 cm	9 cm

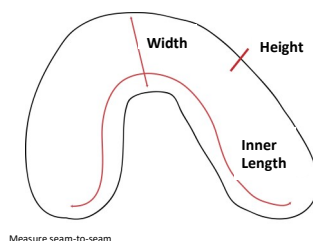

Specifications


Triangular Cushions

Item No.	Name of Product	Colour of outer cover	Length	Width	Height
200	Triangular Cushion, L	Marine blue	94 cm	24,5 cm	22 cm
210	Triangular Cushion, M	Yellow	66 cm	20 cm	17,5 cm
220	Triangular Cushion, S	Blue	39,5 cm	12 cm	10 cm
230	Triangular Cushion, M/S	Red	55 cm	16 cm	14 cm

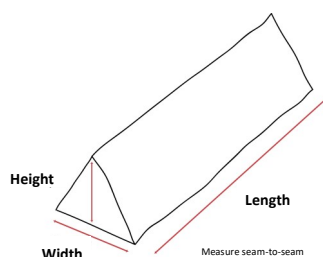

Square Cushions


Item No.	Name of Product	Colour of outer cover	Length	Width	Height
700	Square Cushion, S	Red	59,5 cm	29,5 cm	5,5 cm
710	Square Cushion, L	Green	73 cm	42 cm	5 cm
720	Square Cushion, long	Yellow	110 cm	41 cm	10 cm
730	Square Cushion	Blue	35 cm	22 cm	3 cm
740	Square Cushion, XS	Yellow	40 cm	10 cm	3 cm
750	Square Cushion	Marine	80 cm	40 cm	8 cm

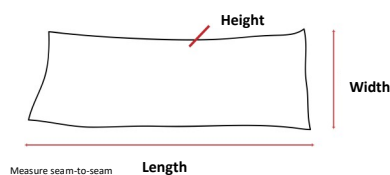

Specifications


Half Moon Cushions

Item No.	Name of Product	Colour of outer cover	Length	Inner Length	Width	Height
300	Half Moon Cushion, S	Blue	47 cm	-	22,5 cm	4 cm
800	Half Moon Cushion, L	Green	-	79 cm	33 cm	7 cm
810	Half Moon Cushion, L & thick	Marine blue	-	86 cm	40 cm	7 cm

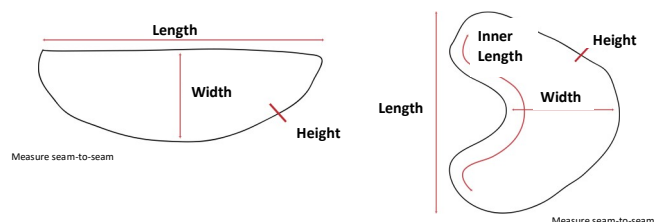

Neck Collar


Item No.	Name of Product	Colour of outer cover	Circumference of Neck hole	Width	Height
310	Neck Collar, Adult	Marine blue	24 cm	21 cm	7 cm
320	Neck Collar, Child	Green	23cm	16 cm	5 cm

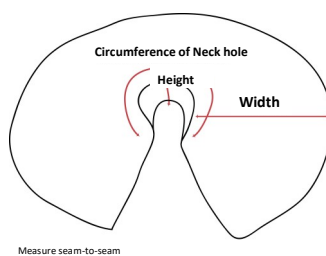

Specifications


Neck Cushions

Item No.	Name of Product	Colour of outer cover	Inner Length	Outer Length & Length	Width	Height
900	Childrens Neck Cushion, S	Red	62 cm	109 cm	33 cm	4,5 cm
910	Junior Neck Cushion, L	Blue	60 cm	114 cm	21 cm	6 cm
920	Junior Neck Cushion, S	Marine	54 cm	90 cm	15 cm	4 cm

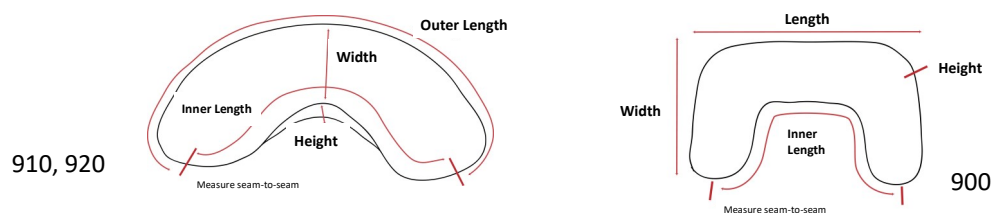

Support Cushions

Item No.	Name of Product	Colour of outer cover	Length	Width	Height A/B
510	Chubby Support Cushion	Blue	34 cm	24 cm	14 / 5,5 cm
400	Sit Cushion, high	Red	73 cm	50 cm	29 / 13 cm
420	Sit Cushion, long	Yellow	104 cm	50 cm	40 / 18,5 cm

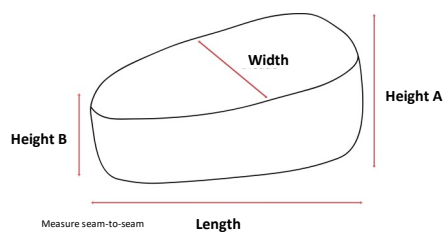

Spare covers can be ordered separately for alle cushions.

Description of materials

Outer cover: 100% ÖKO TEX certified cotton.

Inner cover: PU-covered flame retarded polyester textile.

Filling: Flame retarded EPS beads (Expanded Polystyrene).

Cleaning and maintenance

The outer cover is machine washable at 60° C and can be tumble dried at low temperatures.

The inner cover can be whipped off with alcohol, wipes for chlorine disinfection (max 1000 ppm) or common cleaning agents and water.

The cushion itself must not be chemically cleaned and must not come into contact with any solvent as this could cause decomposition of the e beads.

Disposal

The cushions can be disposed of together with common household waste.

**In spite of high user comfort and maximum pressure distribution
the cushions must not be considered antidecubitus products.**

!!!IMPORTANT!!!

**Avoid contact with sharp edges and
pointed items against the cushions.**



LASAL A/S, Industrivej 15, DK-8881 Thorsoe

E-mail: info@lasal.dk



+45 8696 6666



www.lasal.dk

VAT No. DK 1437 8677