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07/2019/CP

Instructions for XXL-Rehab 4-point Cane

CAUTION

This product must not be used without proper instruction by a qualified professional.

Using the product for individuals whose weight exceeds the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage to the product or personal injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff made without written authorisation from Cobi Rehab.

 To ensure proper use of the XXL-Rehab 4-point Cane, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warnings and safety instructions marked with .

XXL-Rehab 4-point Cane may only be adjusted by qualified staff who have been instructed in the proper use of the cane. Incorrect adjustment can injure the user.

Use

The XXL-Rehab 4-point Cane is a cane specially designed to provide support and safety while walking for bariatric users weighing up to 270 kg.

 If used by someone weighing more than 270 kg, there is a risk that the 4-point Cane will break or sustain damage.

Description

The XXL-Rehab 4-point Cane is supplied with a dual button height adjustment, a rubber handle and a wrist strap.

Before use

The hand support is height adjustable by pressing the dual button height adjustment.

 The XXL-Rehab 4-point Cane may not be used before the dual button height adjustment is pressed fully in place, otherwise there is risk that the inner tube might loosen, causing the user to fall.

Maintenance and cleaning

The XXL-Rehab 4-point Cane can be cleaned with water and ordinary detergent.

The dual button height adjustment should be regularly inspected and if necessary replaced.

⚠ Failure to inspect the the dual button may cause malfunction.

Warranty

When delivered, the XXL-Rehab 4-point Cane is free from defects. Should, contrary to all inspections, there be any defects or damage to the 4-point cane, please contact Cobi Rehab immediately.

The XXL-Rehab 4-point Cane is covered by a 1-year warranty from the date of purchase. The invoice serves as warranty certificate.

The warranty does not apply to:

- products whose serial number, batch number etc. have been removed or considerably damaged
- products that have been repaired by unauthorised staff
- normal wear and tear, for example cleaning, adjustment, lubrication and update of equipment or parts.

The warranty is void if Cobi Rehab assesses that XXL-Rehab 4-point Cane has been used for purposes other than its intended use, for example, if its weight capacity is exceeded. In case of incorrect use, both service and spare parts will be invoiced.

The warranty described above is the only valid warranty. Cobi Rehab may not be held liable for damages of any kind.

XXL-Rehab 4-Point Cane is manufactured and tested in accordance with international standards EN ISO 11334-1:2007.

Specifications

Maximum weight capacity is	270 kg
Height adjustable	71-93 cm
Weight	1.5 kg
Base measurement	15 x 22 cm
Reflex	Yes
Material	Anodized steel
Grip	Vinyl
ISO - classification	ISO 12 03 16 - 03
Expected product service life	7 years

Disposal

Products that can no longer be used are to be disposed of in a sustainable way. Please contact your municipality for information on proper disposal.

MANUFACTURER AND DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
www.cobirehab.com

07/2019/CP

COBI REHAB[®]

The Positioning Company

COBI REHAB
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel. +45 7025 2522
www.cobirehab.com