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07/2019/CP

# Instructions for XXL-Rehab Bariatric walking frame

## CAUTION

This product must not be used without proper instruction by a qualified professional.

Using the product for individuals whose weight exceeds the maximum weight-bearing capacity may pose a risk for damaging the product.

Cobi Rehab assumes no responsibility for any damage to the product or personal injury caused by improper installation, assembly or use of this product.

If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff made without written authorisation from Cobi Rehab.

 To ensure proper use of the XXL-Rehab Bariatric walking frame, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .

XXL-Rehab Walking Frame may only be adjusted by qualified staff who have been instructed in the proper use of the walking frame. Incorrect adjustment can injure the user.

## Use

The XXL-Rehab Walking Frame is designed to provide support, stability safety while walking for bariatric users weighing up to 270 kg.

 If used by someone weighing more than 270 kg, there is a risk of tilting or damaging the product.

## Description

The XXL-Rehab Walking Frame is a foldable walking aid, available in two widths, either with ferrules or with two massive wheels in front. With two wheels in front the walking frame is easy to maneuver and roll across the floor. The two rear legs ensure that walking frame does not roll away if the user loses balance.

## Before use

### Adjusting leg height

1. Adjust height of each leg to the specific needs of the user.
2. Make sure all legs are the same height.
3. When desired height is reached, lock legs into place by aligning buttons into proper holes, as per illustration.

## During use

### Opening walker

1. Grasp one of the handgrips and pull it away from the center of walker.
2. Repeat with the other handgrip. Push the handgrips away from each other until the button release mechanisms clicks into place.

### Folding walker

1. Push downward the button release mechanisms and hold.
2. While holding buttons down, grab nearest handgrip and push toward the center of walker.
3. Repeat for the other handgrip.

 Make sure that the surface is even for an easier walk.

## Maintenance and cleaning

The XXL-Rehab Walking Frame can be cleaned with water and ordinary detergent. Never use organic solvents that could dissolve the grips (such as turpentine, acetone, gasoline, etc).

## SAFETY GUIDELINES PLEASE READ BEFORE USE

- Make sure the walker is level and all legs are adjusted to an equal height.
- Make sure the walker is adjusted correctly to the user's exact height.
- Make sure the locking mechanisms are functioning properly.
- Do not use this walker in pools, showers or other water environments, as this could result in corrosion of parts that lead to damage to the walker and possible injury to the user.
- Always avoid unsafe walker conditions, such as soft, slippery or moving surfaces, uneven paths, stairs and obstacles.
- When transferring user to or from a wheelchair to the walking frame, be sure the wheels of the wheelchair are securely locked.
- Walker tips must be used and be in good condition, replace them whenever needed or missing.

## Specifications

Maximum weight capacity is	270 kg
Total length	50 cm
Total width	74 / 84 cm
Inside width	54 / 64 cm
Adjustable height in 2.5 cm interval	76-101 cm
Weight	3 kg
Foldable	Yes
Frame	Powder coated light metal
Grip	PU foam
Wheels	Massive
ISO - classification	ISO 12 06 03 - 06
Expected product service life	7 years

## Disposal

Products that can no longer be used are to be disposed of in a sustainable way. Please contact your municipality for information on proper disposal.

## Warranty

When delivered, the XXL-Rehab Walking Frame is free from defects. Should, contrary to all inspections, there be any defects or damage to the walking frame, please contact Cobi Rehab immediately.

The XXL-Rehab Walking Frame is covered by a 1-year warranty from the date of purchase. The invoice serves as warranty certificate.

### The warranty does not apply to:

- products whose serial number, batch number etc. have been removed or considerably damaged
- products that have been repaired by unauthorised staff
- normal wear and tear, for example cleaning, adjustment, lubrication and update of equipment or parts.

The warranty is void if Cobi Rehab assesses that XXL-Rehab Walking Frame has been used for purposes other than its intended use, for example, if its weight capacity is exceeded.

In case of incorrect use, both service and spare parts will be invoiced.

The warranty described above is the only valid warranty.

Cobi Rehab may not be held liable for damages of any kind.

XXL-Rehab Walking Frame is manufactured and tested in accordance with international standards EN ISO 11334-1:2007.

### MANUFACTURER AND DEALER

Cobi Rehab  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
Tel.: +45 70 25 25 22  
www.cobirehab.com

07/2019/CP

# COBI REHAB<sup>®</sup>

The Positioning Company

**COBI REHAB**  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
Tel. +45 7025 2522  
[www.cobirehab.com](http://www.cobirehab.com)