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11/2019/CP

# Instructions for XXL-Rehab Rollator King

## CAUTION

**Do not use XXL-Rehab Rollator King without proper instruction from trained staff or from persons who have received instruction in the use of the rollator, as improper adjustment or use may injure the user.**

**Please read these instructions thoroughly before using XXL-Rehab Rollator King. Pay particular attention to the safety instructions marked with .**

**Cobi Rehab assumes no responsibility for any damage to the product or personal injury caused by improper use of the XXL-Rehab Rollator King.**

**If components are damaged or missing, contact Cobi Rehab immediately. Do not use substitute parts.**

**Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.**

## Safety instructions

-  Never sit on the rollator and push off with the feet.
-  Always lock the rollator before sitting down.
-  Never use the rollator on stairs and escalators.
-  Never use the rollator as a wheelchair.
-  Never use the rollator to transport large items – only items that fit in the basket.
-  Never use the rollator on surfaces that slope by more than 3,5 degrees.
-  Never use the rollator on soft and uneven surfaces.
-  Always remember to tighten the hand nuts for height adjustment.
-  If the rollator is used as a resting chair, keep both feet on the ground while seated. This stops the rollator from tipping backwards if the user leans against the backrest.

## Use

XXL-Rehab Rollator King is a bariatric walking support, which can also be used as a resting chair, when the brakes are locked.

 Using the product for individuals weighing more than 325 kg may cause the rollator to tip or becoming damaged.

 Beware of pinching hazards around e.g. the handbrake.

 Ensure left and right handles are level to avoid instability problems.

## Assembling the rollator

In addition to the rollator, the box containing the XXL-Rehab Rollator King also contains two handles (connected to the rollator by cables), a basket and a safety backrest.

1. Unfold the rollator.
2. Remove the screws for height adjustment from the handlebar.
3. Attach the bars to the rollator.
4. Mount the back on the front of the rollator by pulling upwards on the knobs on the left and right sides, respectively. (The knob can be locked to make it easier to mount the back. This is done by pulling the knob upwards and turning it a quarter turn. Remember to loosen the knob again to fasten the safety back rest.)
5. Mount the basket in the front by placing the two hooks on the upper tube below the seat. Place the notch on the bottom of the basket on top of the bottom tube.
6. Adjust the height of the handles to suit the user.
7. Adjust the brakes.

Do not use the rollator without the safety backrest. The purpose of the safety backrest is to ensure that the weight is not positioned too far forward. If the weight is positioned too far forward, the rollator may tip.

## Adjusting the height of the handles

1. Remove the screw and the height-adjustment knob from both handles to enable the handles to move freely up and down.
2. Push the handle up or down until the required height has been found. Reinsert the screw and the knob again. Place the bolt on the back side, so that it fits in the hole.
3. Tighten.

⚠ Ensure left and right handles are level to avoid instability problems.

## How to fold the rollator

1. Grab the front of the seat (the side closest to you when walking) with one hand. Hold one of the handles with the other hand.
2. Lift the seat.
3. The rollator now folds.

To minimize the size of an already folded rollator, remove the back and basket.

## The brakes

The walker can brake in two ways:

1. Brief stops:  
Squeeze both brake handles upwards.
2. Longer stops:  
Push both brake-handles downwards, so that they lock in a “down” position. The handles stay in this position until they are actively released.

## Brake inspection

The brakes are properly set on delivery.

Brake operation, and parking brake should be checked before every use.

If the parking brake cannot lock or if the brakes are not working correctly, contact Cobi Rehab immediately.

Service the brakes every three months.

## Maintenance and cleaning

The frame, seat, handles and safety backrest may be cleaned with water and a household detergent.

Check bolts and nuts every three months. Retighten if required.

⚠ Non-compliance with the bolt inspection guidelines may lead to malfunction.

## Warranty

Cobi Rehab warrants that the XXL-Rehab Rollator King is without defects on delivery. If, against all expectations, the rollator is subject to deficiencies or defects on receipt, please contact Cobi Rehab immediately.

The warranty period for the XXL-Rehab Rollator King is one year from the date of purchase. The invoice if your certificate of warranty.

### The warranty does not apply to:

- Products on which the serial number, batch number or the like have been removed or significantly damaged
- Products which have been repaired by unauthorized personnel
- Normal maintenance such as cleaning, adjustment lubrication and updating of equipment or parts.

The warranty does not apply, if Cobi Rehab assesses that the XXL-Rehab Rollator King has been improperly handled, e.g. if the maximum user weight has been exceeded.

For damage caused by improper operation, an invoice will be issued for both the service and the spare parts used.

The above warranty is the only warranty offered, Cobi Rehab cannot be held liable for damage or injury of any kind.

## Specifications

Maximum user weight	325 kg
Total length	85 cm
Total width	71 cm
Width between handles	61 cm
Height adjustable in 2.5 cm intervals	77.5-99 cm
Seat width	51 cm
Seat depth	35.5 cm
Seat opening	58 cm
Seat height	50 cm
Weight	15.7 kg
Weight capacity in basket	5 kg
Resting chair	Yes
Foldable	Yes
Massive wheels	Yes
Wheels	8" massive
Frame	Powder-lacquered aluminum
Seat/backrest	Polyurethane
ISO - classification	ISO 12 06 06 - 11
Expected product service life	7 years

## Disposal

Products which are no longer used, must be responsibly disposed of. Contact the relevant local authorities for guidance on correct disposal.

XXL-Rehab Rollator King is manufactured and tested in accordance with the international DS/EN 12182:2012 standard.

### MANUFACTURER AND DEALER

Cobi Rehab  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
Tel.: +45 70 25 25 22  
cobirehab.com

11/2019/CP

# **COBI REHAB**<sup>®</sup>

The Positioning Company

**COBI REHAB**  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
Tel. +45 7025 2522  
[cobirehab.com](http://cobirehab.com)