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11/2019/CP

Instructions for XXL-Rehab Stand Tall Walker

CAUTION

Do not use XXL-Rehab Stand Tall Walker without proper instruction from trained staff or from persons who have received instruction in the proper use of the walker as improper adjustment or use may injure the user.

Please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .

Cobi Rehab assumes no responsibility for any damage to the product or personal injury caused by use of XXL-Rehab Stand Tall Walker.

If components are damaged or missing, contact Cobi Rehab immediately. Never use substitute parts.

Cobi Rehab cannot be held liable for defects or accidents caused by repairs carried out without written authorisation from Cobi Rehab.

Use

XXL-Rehab Stand Tall Walker is used in hospitals and nursing homes.

XXL-Rehab Stand Tall Walker provides support and security for users weighing up to 325 kg. The Stand Tall Walker is only suited for indoor use.

 Using the Stand Tall Walker for persons weighing more than 325 kg may cause the Stand Tall Walker to tip or become damaged.

 The Stand Tall Walker should be used on a level, hard surface with a maximum slope of 3.5 degrees. Using the Stand Tall Walker on a surface with a greater slope increases the risk of tipping.

Inspect and service the brakes every six months.

Description

The Bariatric Stand Tall Walker is equipped with four horizontal, through-going bolts (two on each side) for height adjustment of the underarm padding. The bolts are locked with hand nuts.

It is possible to mount an IV holder (or a urine-bag) on the Stand Tall Walker. The IV pole attachment has a length of 101 cm and is attached to the outside of the walking frame holder.

The attachment of the IV pole increases the width of the Stand Tall Walker by max. 16 cm.

The IV holder may only be mounted onto the Stand Tall Walker by authorized staff.

Adjusting the height

The height of the Stand Tall Walker underarm padding is adjusted as follows:

- Screw off the hand nuts
- Pull the two horizontal bolts out of the frame set the underarm padding at the desired position
- Push the bolts through the frame
- Screw on the hand nuts to secure the bolts.

Do not use the Stand Tall Walker until the hand nuts have been screwed tightly onto the bolts, as the underarm padding may otherwise loosen.

Please ensure that the curved heads of the bolts abut closely against the frame, as the hand nuts can otherwise not be tightened correctly.

The hand nuts must face forward on the Stand Tall Walker to avoid injury to the user.

During use

Lock the brakes by bringing the locking bracket over the brake handles, while keeping the Stand Tall Walker braked.

 When using the Stand Tall Walker for support, the brakes must be locked when the user gets up/sits down.

Maintenance and cleaning

The Stand Tall Walker can be cleaned with water and a household detergent.

Lock nuts and bolts must be checked on a regular basis and tightened, if necessary.

⚠ Non-compliance with the bolt inspection guidelines may lead to malfunction.

Specifications

Maximum user weight	325 kg
Total length	73 cm
Total width	76 cm
Width between underarm support, from center to center, adjustable	68-81 cm
Internal width	72 cm
Stand Tall turning radius	2 meters
Height, adjustable in 2,5 cm intervals	100-130 cm
Weight	20 kg
Front wheel	100 mm
Rear wheel	125 mm
Frame	Powder-lacquered chromium molybdenum steel
Grip	PU foam
Underarm support	PU
ISO - classification	ISO 12 06 12 - 06
Expected product life service	7 years

Warranty

Cobi Rehab warrants that the XXL-Rehab Stand Tall Walker is without defects on delivery. If, against all expectations, the Stand Tall Walker is subject to deficiencies or defects on receipt, please immediately contact Cobi Rehab.

The warranty period for the XXL-Rehab Stand Tall Walker is one year from the date of purchase. The invoice is your certificate of warranty.

The warranty does not apply to:

- products on which the serial number, batch number or the like have been removed or significantly damaged
- which have been repaired by unauthorized personnel
- normal maintenance such as cleaning, adjustment, lubrication and updating of equipment or parts.

The warranty does not cover when Cobi Rehab assesses that XXL-Rehab Stand Tall Walker has been improperly handled, e.g. if the maximum user weight has been exceeded.

For damage caused by improper operation, an invoice will be issued for both the service and the spare parts used.

The above warranty is the only warranty offered. Cobi Rehab cannot be held liable for damage or injury of any kind.

Disposal

Products which are no longer used, must be responsibly disposed of. Contact the relevant local authorities for guidance on correct disposal.

XXL-Rehab Stand Tall Walker is produced and tested in compliance with international standards DS/EN ISO 11199-3:2005 and DS/EN 12182:2012.

MANUFACTURER AND DEALER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
cobirehab.com

11/2019/CP

COBI REHAB[®]

The Positioning Company

COBI REHAB
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel. +45 7025 2522
cobirehab.com