



# Cobi Positioning Cushions

## Cleaning and disinfection guide

Correct cleaning routines and maintenance can extend the lifespan of the product.



### Daily cleaning of cushions

- 1, Remove dirt from the cushion.
- 2, Wipe the cushion with a clean cloth wrung out in water with a mild detergent.

3, Wipe the cushion thoroughly with a clean, dry cloth. The cushion must be completely dry before used again or stored up.

### Disinfection and wash

For surface disinfection of cushions, disposable cloths with ethanol (e.g. Wet Wipe 70%) or a chlorine product 1000-12000 ppm (Wet Wipe Chlorine disinfection) can be used.

Accessories such as hygiene and jersey covers can be washed at 60° C and tumble dried.

When the different types of covers are machine washed, the zipper must be zipped up as it otherwise may be damaged.

Cobi Positioning Cushions cannot be washed. Machine wash can destroy the filling of the cushions.