



Lasal Positioning Cushions

Cleaning and disinfection guide

Correct cleaning routines and maintenance can extend the lifespan of the product.



Daily cleaning of cushions

- 1, Remove dirt from the cushion.
- 2, Wipe the cushion with a clean cloth wrung out in water with a mild detergent.

- 3, Wipe the cushion thoroughly with a clean, dry cloth. The cushion must be completely dry before used again or stored up.

Disinfection and wash

For surface disinfection of cushions, disposable cloths with ethanol (e.g. Wet Wipe 70%) or a chlorine product 1000-12000 ppm (Wet Wipe Chlorine disinfection) can be used.

Hygiene cover and white cotton cover can be washed at 95° C and tumble dried.

Colored cotton covers can be washed at 60° C and tumble dried. Note that the colored covers can fade over time.

Lasal Positioning Cushions cannot be washed. Machine wash can destroy the filling of the cushions.

When the different types of covers are machine washed, the zipper must be zipped up as it otherwise may be damaged.