



## PRODUCT SHEET

# Cobi Positioning Cushions

## Improving the quality of life of immobile patients

Cobi Positioning Cushions are developed exclusively for positioning of patients within the following patient groups: bariatric, dementia, geriatric, neurology, palliation, and chronic pain patients.

Used in e.g., care centers, institutions, hospices and in private homes.



- Pressure relieving effect.
- Anti-bacterially treated cushion cover.
- Many applications and great flexibility.
- Two different types of washable covers.
- Provides tactile stimuli.
- Developed by Cobi Rehab in cooperation with special units within dementia and neurology.



Scan or click on the code and  
find the product at the  
AssistDATA.



Cobi Rehab and Cobi Rehab suppliers comply with the UN conventions  
on child labour and forced labour as well as ILO 029, C132 and C182.

## Use

Cobi Positioning Cushions helps solving different positioning issues, meeting every patients' specific needs. All cushions are developed by Cobi Rehab in cooperation with special units within dementia and neurology.

Each cushion is suitable for the following patient groups: bariatric, dementia, geriatric, neurology, palliation, and chronic pain patients. These patient groups have in common the high risk of developing tissue damage and pressure ulcers.

### Recommendations for the best positioning

Stability and filling of cavities are very important when ensuring optimal positioning. Stability and filling of cavities distribute the pressure over an area as wide as possible. In this way, the user remains in the desired, natural position, without any resistance in the joints, nerve and/or muscle tissue.

### Construction

In the design of Cobi Positioning Cushions, we have focused on being able to position the cushions tightly and close to the user's body.

The cushions have a certain weight which contributes to staying in place once positioned and provide both stability and support.

The cushions are filled with 3mm Krøyer pearls, PU foam, comfort foam, granules, and 5mm pearls of expanded polypropylene. Distribution and content are adapted to each individual cushion.

There are two types of covers, a hygiene cover, and a jersey cover. Both covers are breathable, washable, anti-bacterially treated and stretchable.

### Soothing effect

In addition to reducing pressure and increasing pressure distribution, the cushions are particularly suitable for reducing spasticity, as well as providing tactile stimuli, which is recommended in cases of e.g., dementia.

Being positioned and surrounded by cushions provides a sense of being cocooned, which often has a calming effect and increases comfort.

### Cleaning and maintenance

#### Daily cleaning

Wipe the cushions with a clean cloth wrung out in water with a mild detergent. The cushions must be completely dry before they are used again or put away.

#### Desinfection and wash

For surface disinfection of the cushions use disposable cloths with ethanol (e.g., Wet Wipe 70



Cobi U-cushion under the upper body. Cobi Cylinder cushion short under the knees. Cobi Fin cushion under the lower calves.



Large Cobi cushion under lower legs.



Cobi Flexi cushion along the front of the body.

%) or a chlorine product 1000-1200 ppm (e.g., Wet Wipe Chlorine Desinfection).

Accessories such as hygiene covers, and jersey covers can be washed at 60° C and tumble dried.

Cobi Positioning Cushions does not withstand machine washing. Machine washing will destroy the filling materials.

When machine washing the covers, the zipper must be zipped, otherwise it may be damaged.

## Cobi Positioning Cushions



Cobi Fin cushion



Cobi H-cushion



Cobi Cylinder cushion short




Cobi Seahorse Junior cushion



Cobi Small pillow



Cobi Large pillow

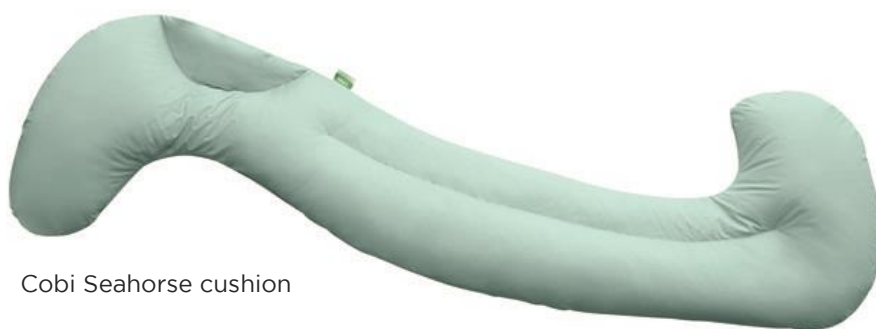

Cobi Seahorse cushion



Cobi U-cushion



Cobi Universal cushion



Cobi Flexi cushion



Cobi Robust cushion



Cobi Lap cushion



Cobi Heel/arm cushion



Cobi Wedge cushion

---

You can always contact Cobi Rehab on telephone +45 7025 2522  
or e-mail [cobi@cobi.dk](mailto:cobi@cobi.dk) if you want more information about Cobi Positioning  
Cushions.

---

## Specifications

Filling materials	3mm Krøyer pearls of expanded polystyrene, 5mm pearls of expanded polypropylene, granules, foam material (PU foam and comfort foam). Distribution and content are adapted to each cushion type.
Cushion surface	Breathable and waterproof. Polyamide impregnated with PU and anti-bacterially treated.
Hygiene cover	Breathable and waterproof. Polyamide impregnated with PU and anti-bacterially treated. Flame retardant.
Jersey cover	Cotton (ØKO-TEX® Standard 100) and modacrylic.
Cleaning	Both covers are washable and can be tumble dried. Cobi Positioning Cushions does not withstand machine washing but can be cleaned with a mild detergent or disinfected.
Storage	Do not expose the cushions to high temperatures, prolonged and intense sunlight, or other radiation.
ISO classification	ISO 09 07 06 - 02
Expected product service life	2 years
Basic UDI-DI	5740000100014N5

## Ordering

Model	Length	Width	Height	Weight	Item number
Cobi Cylinder Cushion short	75 mm	25 mm	25 mm	1.7 kg	0400-064-000
Cobi Flexi cushion	194 mm	27 mm	15 mm	3.7 kg	0400-068-000
Cobi Robust cushion	200 mm	28 mm	28 mm	6 kg	0400-070-000
Cobi Heel/arm cushion	40 mm	28 mm	11 mm	0.5 kg	0400-060-000
Cobi Universal cushion	58 mm	39 mm	22 mm	1.7 kg	0400-054-000
Cobi Seahorse cushion	186 mm	51 mm	12 mm	5 kg	0400-056-000
Cobi Seahorse Junior cushion	127 mm	27 mm	12 mm	2.5 kg	0400-057-000
Cobi Fin cushion	64 mm	34 mm	8 mm	0.6 kg	0400-058-000
Cobi U-cushion	88 mm	94 mm	17 mm	2.7 kg	0400-072-000
Cobi Lap cushion	55 mm	54 mm	12 mm	1.2 kg	0400-050-000
Cobi Small pillow	39 mm	25 mm	8 mm	0.3 kg	0400-048-000
Cobi Large pillow	58 mm	40 mm	13 mm	0.9 kg	0400-049-000
Cobi H-cushion	51 mm	30 mm	30 mm	1.8 kg	0400-074-000
Cobi Wedge cushion	40 mm	50 mm	14 mm	1.1 kg	0400-066-000
Cobi Arm cushion, left	20 mm	7 mm	60 mm	0.9 kg	0400-052-000V
Cobi Arm cushion, right	20 mm	7 mm	60 mm	0.9 kg	0400-052-000H