



PRODUCT SHEET

Lasal Positioning Cushions

The solution to positioning.

Lasal Positioning Cushions are developed exclusively to the positioning of users within the following patient categories: dementia, geriatrics, neurology, palliation, and chronic pain patients.

Lasal Positioning Cushions are used in care centers, institutions, hospices, and private homes.



- Pressure-distributing effect.
- Anti-bacterially treated cushion cover.
- Mouldable and adapts to the posture.
- Washable covers.
- Provides tactile and proprioceptive stimuli.
- More than 25 different cushions in the series.



Use

With a Lasal Positioning Cushion the user will experience a comfortably and pleasant support of the body without feeling fixed.

For the caregiver, the cushions are an easy-to-use tool facilitating more varied and correct postures. Correct postures can prevent formation of contractures, reduce spasticity, and prevent pressure ulcers.

Lasal Positioning Cushions are also ideal for delimitation and increased body-consciousness.

The cushions are used to position adults and children in supine and sitting positions.

Please notice, that the cushions are not a proven pressure ulcer product despite the high comfort and pressure distribution.

Construction

Lasal Positioning Cushions are available in multiple shapes and sizes. The combination of small beads of polystyrene in a flexible inner cushion covered with a hygiene or cotton cover contributes to each cushion adjusting to the body shape as well as the supporting surface.

Lasal Positioning Cushions are mouldable and therefore more stable than ordinary cushions.

There are two different covers to each Lasal Positioning Cushion. A cotton cover in different colors and a grey hygiene cover.

The cotton cover is breathable, washable, and anti-bacterially treated. The hygiene cover is washable and mold and fungus resistant.

Soothing effect

In addition to reducing pressure and increasing pressure distribution, the cushions are particularly suitable for reducing spasticity, as well as providing tactile stimuli, which is recommended in cases of e.g., dementia.

Being positioned and surrounded by cushions provides a sense of being cocooned, which often has a calming effect and increases comfort.

Cleaning and maintenance

Daily cleaning

Wipe the cushions with a clean cloth wrung out in water with a mild detergent. The cushions must be completely dry before they are used again or put away.

Desinfection and wash

For surface disinfection of the cushions use disposable cloths with ethanol (e.g., Wet Wipe 70 %) or a chlorine product 1000-1200 ppm (e.g., Wet Wipe Chlorine Desinfection).



Banana Cushion, L, along the abdomen and between the thighs. Square Cushion, L, between the lower leg.



Banana Cushion, S, behind the back and under the arm. Banana Cushion, L, in front of the body and between the thighs. Square Cushion, L, between the lower legs.



Half Moon Cushion, thick, in the lap. Half Moon Cushion behind the neck. T-Cushion, M, between the thighs.

The cotton cover is machine washable at 60° C and the hygiene cover at 95° C. Both covers can be tumble dried at low temperatures.

The cotton cover shrinks approx. 5% and the color will fade over time.

Lasal Positioning Cushions without cover do not withstand machine washing but can be cleaned with a mild detergent or disinfected.

Lasal Positioning Cushions



U-shaped Neck Cushion, child, S
Size 62/109x33x4.5 cm
Item number: 0305-901-000

Neck Cushion, child
Size 23x16x5 cm
Item number:
0305-321-000



Half Moon Cushion, S
Size 47x22.5x4 cm
Item number:
0305-301-000



Half Moon Cushion
Size 79x33x7 cm
Item number:
0305-801-000



Half Moon Cushion, thick
Size: 86x40x7 cm
Item number:
0305-811-000



U-shaped Neck Cushion, junior, S
Size 54/90x15x4 cm
Item number: 0305-921-000



Neck Cushion, adult
Size 24x21x7 cm
Item number:
0305-311-000

Triangular Cushion, M/S
Size 55x16x14 cm
Item number:
0305-231-000



U-shaped Neck Cushion, junior, L
Size 60/114x21x6 cm
Item number:
0305-911-000



Triangular Cushion, L
Size 94x24.5x22 cm
Item number:
0305-201-000



Triangular Cushion, M
Size 66x20x17.5 cm
Item number:
0305-211-000



Triangular Cushion, S
Size 39.5x12x10 cm
Item number:
0305-221-000

T-Cushion, XS
Size 36/13x27/12.5x4 cm
Item number: 0305-6041-000



T-Cushion, S
Size 56.5/17.5x37/15x7 cm
Item number: 0305-6031-000



T-Cushion, M
Size 63/22x40/19.5x7 cm
Item number: 0305-6021-000



T-Cushion, L/M
Size 72/25x45/20x8 cm
Item number: 0305-6016-000



T-Cushion, L, Size 83/27x56/27x10 cm
Item number:
0305-6011-000



T-Cushion, XL, Size 98/26x61/29x11 cm
Item number:
0305-6001-000



Support Cushion
Size 34x24x14/5.5 cm
Item number:
0305-511-000

Sit Cushion, long
Size 104x50x40/18.5 cm
Item number:
0305-421-000



Sit Cushion, high
Size 73x50x29/13 cm
Item number:
0305-401-000

Square Cushions, L
Size 73x42x5 cm
Item number:
0305-711-000



Square Cushions, Long
Size 110x41x10 cm
Item number:
0305-721-000

Square Cushions, S
Size 59.5x29,5x5.5 cm
Item number:
0305-701-000



Square Cushions, XS
Size 40x10x3 cm
Item number:
0305-741-000



Square Cushions
Size 35x22x3 cm
Item number:
0305-731-000



Snake Shaped Cushion, Size 320x30x9 cm
Item number:
0305-151-000

Banana Cushion, S
Size 144x40x12 cm
Item number: 0305-111-000

Banana Cushion, L, Size 148x31x10 cm
Item number:
0305-101-000

Specifications

Filling	Krøyer-beads of expanded polystyrene. Flame retardant.
Inner cover	PU-covered flame retarded polyester textile. Mold and fungus resistant.
Cotton cover	100% Öko-Tex certified cotton. Anti-bacterially treated.
Hygiene cover	PU-covered flame retardant polyester textile. Mold and fungus resistant.
Cleaning of inner cover	The inner cover does not withstand machine washing but can be cleaned with a mild detergent or disinfected.
Washing of covers	The cotton cover is machine washable at 60° C and the hygiene cover at 95° C. Both covers can be tumble dried at low temperatures. The cotton cover shrinks approx. 5% and the color will fade over time.
Storage	Do not expose the cushions to high temperatures, prolonged and intense sunlight, or other radiation.
ISO-classification	09 07 06 - 02
Expected product service life	2 years
Basic UDI-DI	5740000100014N5
HS code	94049090

You can always contact Cobi Rehab on +45 7025 2522
or cobi@cobi.dk for more information about
Lasal Positioning Cushions.

Cobi Rehab | Fuglebækvej 1D, DK-2770 Kastrup, Denmark | cobi@cobi.dk | cobirehab.com

Ordering Lasal Positioning Cushions with cover

On the overview page 3 you find all our Lasal Positioning Cushions with their item numbers. Each item number only applies to the inner cushion itself. Item numbers on hygiene covers end with 010, while cotton covers end with 020.

How to order cushions and covers:

Square Cushion, Long, Item number 0305-721-000

Square Cushion, Long, Hygiene cover, Item number 0305-721-010

Square Cushion, Long, Cotton Cover, Item number 0305-721-020

Order your Lasal Positioning Cushions at cobi@cobi.dk.
