



COBI REHAB[®]
The Positioning Company

USER MANUAL

Bariatric Comfort Wheelchair Cushion

Danish designed standard wheelchair cushion to prevent pressure relief for bariatric users.

Comfort Wheelchair Cushion is intended for users weighing up to 325 kg.



Scan or click the QR code and find the product on the Danish AssistData.



Cobi Rehab and Cobi Rehab's supplier comply with UN conventions on child and forced labor and ILO 029, C132, and C182.

Table of content


1. Safety	3
2. Symbols	4
3. General comments	5
4. Use	5
5. In use	5
6. Long-term storage.....	6
7. Cleaning and maintenance	6
8. Disposal.....	6
9. Warranty	7
10. Specifications	8
11. Ordering.....	8

1. Safety



WARNING

Do not use the Comfort Wheelchair Cushion without prior thorough instructions by trained staff or by persons who have received instructions in the use of the Comfort Wheelchair Cushion. Improper use may cause injury to the user.

Carefully study these instructions before using the Comfort Wheelchair Cushion. Pay attention to Safety Instructions indicated by a warning triangle .

Cobi Rehab accepts no responsibility for damage caused to the product nor for personal injury caused by improper use of the Comfort Wheelchair Cushion.

Contact Cobi Rehab immediately in case of damaged or missing parts. Never use your own spare parts.

Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.



SAFETY INSTRUCTIONS

Using Comfort Wheelchair Cushion for persons weighing more than 325 kg may cause damage to the cushion or the user.




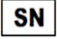


Do not place any further layers or padding between the cushion and the user as this may reduce the effect of the cushion.

A wet or defective Comfort Wheelchair Cushion can reduce the effect of preventing of the pressure relief or in worse case cause personal injury.

Always disinfect the Comfort Wheelchair Cushion before transferring to a new user. This should be done to avoid cross contamination.

Lack of maintenance can damage the product, shorten the product service life, or reduce the performance of Comfort Wheelchair Cushion.

2. Symbols

The following symbols are used in this manual and/or on the product label.	
	Caution!
	Declaration of conformity, Medical Device Regulation, EU 2017/745
	Cobi Catalogue number
	Serial number
	Date of manufacture
	Manufacturer
CE-Label is located on the packaging. See example of the label below.	



3. General comments

This user manual contains important information about the target group and the use of Comfort Wheelchair Cushion. It also includes a list of specifications.

The most recent version of this user guide can be found on our website. Check if the version you are reading now is the most recent.

If you need a user manual that is written in a larger font size, the guide is available in PDF format at [cobirehab.com](https://www.cobirehab.com), which can be scaled up to the required font size.

You can always contact Cobi Rehab on + 45 70252522 or cobi@cobi.dk if you want more information about Comfort Wheelchair Cushion.

Cobi Rehab reserves the right to change the product specification without providing prior notice.

Comfort Wheelchair Cushion's basic UDI number can be found in the specification list.

Product safety notices and product recalls can be viewed on the website.

<https://www.cobirehab.com/kundehoscobi/vigtig-produktinformation/>

4. Use

Comfort Wheelchair Cushion is a standard wheelchair cushion for the prevention of pressure relief for bariatric users.

Comfort Wheelchair Cushion consists of a three cm viscoelastic foam layer at the top. The base is made of a Heavy Duty PUR foam with springs that cut into the under base of the foam.

The string system is patented and ensures that the pressure is distributed to a larger surface, which increases the pressure relief.

The included incontinence cover is made of Dartex material. The incontinence cover is breathable, antibacterial and fire retardant.

⚠ Using Comfort Wheelchair Cushion for persons weighing more than 325 kg may cause damage to the cushion or the user.

Comfort Wheelchair Cushion is used in wheelchairs in hospitals, in nursing homes, and private homes.

Always consult a therapist before assigning a Comfort Wheelchair Cushion to a user.

Contraindications

There are no contraindications to the use of the product.

5. In use

Before use

Place the Comfort Wheelchair Cushion in a wheelchair or in a chair so that the zipper faces downwards. Check that the cover on the Comfort Wheelchair Cushion is zipped correctly.

⚠ Do not place any further layers or padding between the cushion and the user as this may reduce the effect of the cushion.

In use

Check the wheelchair cushion regularly to make sure that the cushion and cover are facing correctly and that the cushion is not wet or defective.

⚠ A wet or defective Comfort Wheelchair Cushion can reduce the effect of preventing of the pressure relief or in worse case cause personal injury.

6. Long-term storage

Comfort Wheelchair Cushion should be stored dark at temperatures between -10° - 50° C.

7. Cleaning and maintenance

Comfort Wheelchair Cushion cannot be washed in a washing machine.

The incontinence cover can be cleaned with water and a regular detergent. The cover can also be machine washed at 95 ° C and tumble dried.

Disinfection

Comfort Wheelchair Cushion can also be disinfected with e.g., Wet Wipe 70% or a chlorine product 1000-1200 ppm (e.g., Wet Wipe Chlorine Desinfection).

⚠ Always remember to disinfect Comfort Wheelchair Cushion when transferred to a new user. This is done to avoid cross-contamination and infections.

Local cleaning and disinfection procedures must always be followed.

8. Disposal

Products that can no longer be used are to be disposed of in a sustainable way.

Contact Cobi Rehab or relevant authorities for information on proper disposal.

9. Warranty

Cobi Rehab warrants that the Comfort Wheelchair Cushion is without defects on delivery. If, against all expectations, the wheelchair is subject to deficiencies or defects on receipt, please contact Cobi Rehab immediately.

The warranty period for the Comfort Wheelchair Cushion is one year from the date of purchase and covers manufacturing defects or defect on receipt. The invoice is your certificate of warranty.

The warranty does not apply to:

- products on which the serial number, batch number or the like have been removed or significantly damaged.
- products which have been repaired by unauthorised personnel.

9.1 Unintended event

Report any incidents relating to the use of Comfort Wheelchair Cushion to Cobi Rehab.

9.2 After expiry of expected service life

After the expected service life, it is important that an overall assessment of the product is made before continued use.


After the expected service life, Cobi Rehab cannot guarantee the suitability and safety of the product, as Cobi Rehab has no control over how the product is used and wear and tear.

The overall assessment of the product must be carried out by qualified professionals, and as a

The warranty does not cover:
- wearing parts.

This warranty does not apply when Cobi Rehab assesses that the Comfort Wheelchair Cushion has been improperly operated.

For damage caused by improper operation, an invoice will be issued for both the service and the spare parts used.

 Lack of maintenance can damage the product, shorten the life, or affect the performance of the Comfort Wheelchair Cushion.

10. Specifications

Maximum user weight	325 kg
Weight	2 kg
Seat width	550/610/660/720/820 mm
Seat depth	450 mm
Seat height	120 mm
Base cushion	HD PUR foam and visco elastic PUR foam
Incontinence cover	Dartex
ISO classification	18 10 06 01
Expected product service life	3 years
Risk class	1
Basic UDI	5740000100011MX
The surface temperature of the Comfort Wheelchair Cushion can increase or decrease when exposed to external sources of heat or cold.	

11. Ordering

Model	Width cm	Depth cm	Height cm	Item number
Comfort Cushion with incontinence cover	55	45	12	0230-055-045
Comfort Cushion with incontinence cover	61	45	12	0230-061-045
Comfort Cushion with incontinence cover	66	45	12	0230-066-045
Comfort Cushion with incontinence cover	72	45	12	0230-072-045
Comfort Cushion with incontinence cover	82	45	12	0230-082-045
Incontinence cover	55			0230-055-001
Incontinence cover	61			0230-061-001
Incontinence cover	66			0230-066-001
Incontinence cover	72			0230-072-001
Incontinence cover	82			0230-082-001



COBI REHAB[®]
The Positioning Company



MANUFACTURER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
+45 7025 2522
cobirehab.com