



PRODUCT SHEET

Rollator Support

Strong and functional rollator designed for the bariatric user.

Rollator Support is intended for users weighing up to 200 kg.

The rollator can be used outdoors and indoors in institutions, care centers, and in private homes. Rollator Support comes in two sizes, 61 cm and 71 cm width.



- Fits through most doorways
- Also works as chair for resting
- Extremely strong construction
- Weight capacity up to 200 kg



Rollator Support

General comments

This product sheet contains important information about the use of Rollator Support and the target group for the rollator. It also includes a range of specifications on Rollator Support.

The most recent version of this product sheet can be found on our website. Check if the version you are reading now is the most recent.

If you need a product sheet written in a larger font size, the manual is available in PDF format at

cobirehab.com, which can be scaled up to the required font size.

You can always contact Cobi Rehab on + 45 7025 2522 or cobi@cobi.dk if you want more information about Rollator Support.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Product safety notices and product recalls can be viewed on the website.

Use

Rollator Support is intended for users weighing up to 200 kg. The rollator is designed for use outside and inside, in institutions, care centers and private homes.

The height adjustable forearm supports assist the user to drive the rollator forwards in an upright position and at the same time relieve the wrists.

Rollator Support can be used as a resting chair when the brakes are locked.

Do not use Rollator Support on soft and uneven surfaces.

Contraindications

The user of Rollator Support must have physical and cognitive resources to control and operate (e.g., to walk, steer, brake) the rollator in a safe manner.

Sizes and measures

Rollator Support is available in two sizes, 61cm and 71 cm which refers to the distance between the handles. Both models have a seat width of 51cm, seat height of 51cm and a seat depth of 37 cm.

The total width of Rollator Support is 13 cm more than the distance between the handles. Be aware of the model with a width of 84 cm can have difficulties going through certain doorways.

How to choose the right size

Which size of Rollator Support depends on the body shape and measures of the user. For users with a lot of fullness around the pelvis and thighs, we recommend a size 71 cm.

Contact Cobi Rehab at cobi@cobi.dk for guidance and instructions on how to choose the right size.

Never use the rollator on surfaces that slope by more than 3.5 degrees.

Adjustments

Adjusting the height of the underarm support

Adjusting the height of the underarm support can be done by loosening the knob for height adjustment. Push the screw, so the bars can move freely up and down.

Then push the underarm support up or down until the required height has been found. Reinsert the screw and the knob again.

Safety back rest

When the user is resting on the seat, the back rest ensures the weight of the user is evenly distributed on the seat. This way the risk of tipping is prevented significantly.

Cleaning and disinfection

The frame, seat, handle, and safety back rest of the rollator must be cleaned with water and a regular cleaning agent.

Long term storage

The Rollator Support should be stored in the dark and covered with a sheet or other cover.

Transport in motor vehicle

Rollator Support can easily be folded if the rollator is to be transported in a motorized vehicle.

Grasp the front edge of the seat with one hand. Hold one of the handles with the other hand.

Lift up on the seat and fold the rollator.

Folding the rollator

The rollator can be folded by grabbing the front of the seat (the side closest to user) with one hand. Hold one handle with the other hand. Lifting the seat makes the rollator fold.

Brakes

The rollator brakes are used in two ways. For brief stops and longer stops.

It also makes the user more comfortable when sitting.


Disinfection

The rollator's frame, seat, handle, and safety backrest can also be disinfected with ethanol 70-85% or a chlorine product 1000-1200 ppm .

The upholstered seat and the safety back rest can be exchanged when switching to a new user.

The Rollator Support should be stored at temperatures between -10° and 50°C.

The basket and the safety back rest can be removed to take up less space.

 The Rollator Support must not be used as a seat in a motor vehicle.

Specifications Model 610 mm and 710 mm	
Maximum user weight	200 kg
Total length	850 mm
Total width	740/840 mm
Width between handles	610/710 mm
Height adjustable in 25 mm intervals	100 - 130 mm
Seat width	510 mm
Seat depth	370 mm
Seat opening	610/710 mm
Seat height	510 mm
Weight	17 kg
Weight capacity in basket	5 kg
Foldable	Yes
Massive wheel	Yes
Frame	Powder-lacquered steel/SAE41-30
Frame and seat	Flame retardant
Seat and back	Polyurethane
Wheel	8" massive
Expected product life	7 years
ISO classification	ISO 12 06 06 - 11
Risk class	I
Basic UDI	5740000100007N8
The surface temperature of the Rollator Support can increase or decrease when exposed to external sources of heat or cold.	

Model	Total width	Seat opening	Item number
Rollator Support 610 mm, including basket	740 mm	610 mm	0173-050-000
Rollator Support 710 mm, including basket	840 mm	710 mm	0173-071-000