



PRODUCT SHEET

# Bariatric Comfort Wheelchair Cushion

**Danish designed standard wheelchair cushion providing pressure relief for bariatric users.**

Comfort Wheelchair Cushion is intended for users weighing up to 325 kg.



- Comes in several sizes
- Designed by Cobi Rehab
- Different top and base layer
- Distributes pressure to a larger surface
- Weight capacity up to 325 kg



## General information

This product sheet contains important information about the use of Bariatric Comfort Wheelchair Cushion and the target group for the product. It also includes a range of specifications on Bariatric Comfort Wheelchair Cushion.

The most recent version of this product sheet can be found on our website. Check if the version you are reading now is the most recent.

If you need a product sheet written in a larger font size, the guide is available in PDF format at [cobirehab.com](http://cobirehab.com), which can be scaled up to the required font size.

## Use

Comfort Wheelchair Cushion is a standard wheelchair cushion providing pressure relief for bariatric users.

Comfort Wheelchair Cushion is used in wheelchairs in hospitals, in nursing homes, and private homes.

Comfort Wheelchair Cushion consists of a three cm viscoelastic foam layer at the top. The base is made of a Heavy Duty PUR foam with springs that cut into the under base of the foam.

The string system ensures that the pressure is distributed to a larger surface, which increases the pressure relief.

## Long term storage

Comfort Wheelchair Cushion must be stored dark at temperatures between -10° and 50° C.

## Cleaning and maintenance

Comfort Wheelchair Cushion cannot be washed in a washing machine.

The incontinence cover can be cleaned with water and a regular detergent. The cover can also be machine washed at max 95°C and tumble dried.

You can always contact Cobi Rehab on + 45 7025 2522 or [cobi@cobi.dk](mailto:cobi@cobi.dk) if you want more information about Bariatric Comfort Wheelchair Cushion.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Product safety notices and product recalls can be viewed on the website.

The included incontinence cover is made of Dartex material. The incontinence cover is breathable, antibacterial and fire retardant.

## Settings

Comfort Wheelchair Cushion is placed in a wheelchair or in a chair with the zipper facing downwards.

## Contraindications

There are no contraindications to the use of the product.

## Disinfection

Comfort Wheelchair Cushion can also be disinfected with e.g., Wet Wipe 70% or a chlorine product 1000-1200 ppm (e.g., Wet Wipe Chlorine Desinfection).

# Specifications

Maximum user weight	325 kg
Weight	2 kg
Seat width	550/610/660/720/820 mm
Seat depth	450 mm
Seat height	120 mm
Base cushion	HD PUR foam and visco elastic PUR foam
Incontinence cover	Dartex
ISO classification	18 10 06 01
Expected product service life	3 years
Risk class	I
Basic UDI	5740000100011MX
The surface temperature of the Comfort Wheelchair Cushion can increase or decrease when exposed to external sources of heat or cold.	

# Ordering

Model (+ depth + height in mm)	Width	Item number
Comfort Cushion with incontinence cover (450 + 120 mm)	55	0230-055-045
Comfort Cushion with incontinence cover (450 + 120 mm)	61	0230-061-045
Comfort Cushion with incontinence cover (450 + 120 mm)	66	0230-066-045
Comfort Cushion with incontinence cover (450 + 120 mm)	72	0230-072-045
Comfort Cushion with incontinence cover (450 + 120 mm)	82	0230-082-045
Incontinence cover	55	0230-055-001
Incontinence cover	61	0230-061-001
Incontinence cover	66	0230-066-001
Incontinence cover	72	0230-072-001
Incontinence cover	82	0230-082-001



**COBI REHAB**<sup>®</sup>  
The Positioning Company

---



**MANUFACTURER**

**Cobi Rehab**  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
+45 7025 2522  
[cobirehab.com](http://cobirehab.com)