



PRODUCT SHEET

# Bariatric Walking Cane

**Strong and functional walking cane designed for the bariatric user.**

Bariatric Walking Cane is intended for users weighing up to 270 kg.

Walking Cane can be used outdoors and indoors in institutions, care centers and in private homes.



- Height adjustable
- Designed by Cobi Rehab
- Provide support and safety while walking
- Strong profile construction
- Weight capacity up to 270 kg



## General information

This product sheet contains important information about the use of Bariatric Walking Cane and the target group for the product. It also includes a range of specifications on the Bariatric Walking Cane.

The most recent version of this product sheet can be found on our website. Check if the version you are reading now is the most recent.

If you need a product sheet written in a larger font size, the guide is available in PDF format at [cobirehab.com](http://cobirehab.com), which can be scaled up to the required font size.

## Use

Bariatric Walking Cane is an extra reinforced walking cane designed to provide support and safety while walking for users up to 270 kg.

Bariatric Walking Cane is used at hospitals, care centers and in private homes. Bariatric Walking Cane should only be used on level ground and accessible terrain, indoors as well as outdoors.

## Settings

Bariatric Walking Cane is supplied with two release buttons on the shaft for height adjustment.

Before use, the height of Bariatric Walking Cane must be adjusted by pressing the two release buttons.

## Long term storage

Bariatric Walking Cane should be stored at temperatures between -10° and 50°C.

## Cleaning and maintenance

Bariatric Walking Cane can be cleaned with water and ordinary detergent.

The dual button height adjustment should be regularly inspected and if necessary, replaced.

You can always contact Cobi Rehab on + 45 7025 2522 or [cobi@cobi.dk](mailto:cobi@cobi.dk) if you want more information about Bariatric Walking Cane.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Product safety notices and product recalls can be viewed on the website.

## Contraindications

The user of Bariatric Walking Cane must have physical and cognitive resources to operate the cane in a safe manner.

## Disinfection

Bariatric Walking Cane can be disinfected with ethanol 70-85% or a chlorine product 1000-1200 ppm.

# Specifications

---

Maximum user weight	270 kg
Height adjustable	710-930 mm
Weight	0.9 kg
Reflex on grip	Yes
Material	Chrome molybdenum steel /SAE41-30
Grip	TPR- Thermoplastic rubber
ISO classification	ISO 12 03 03 - 05
Expected product service life	7 years
Risk class	I
UDI-DI	5740000102810
Basic UDI	5740000100010MV
The surface temperature of Bariatric Walking Cane can increase or decrease when exposed to external sources of heat or cold.	

# Ordering

---

<b>Model</b>	<b>Height adjustable</b>	<b>Item number</b>
Bariatric Walking Cane	710-930 mm	0200-000-000



**COBI REHAB<sup>®</sup>**  
The Positioning Company

---



**MANUFACTURER**

**Cobi Rehab**  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
+45 7025 2522  
[cobirehab.com](http://cobirehab.com)