



PRODUCT SHEET

Bariatric Elbow Crutch

Elbow Crutch with extra reinforcement, specially designed to provide support and safety while walking for bariatric users weighing up to 325 kg.

The crutch can be used outdoors and indoors in institutions, care centers and in private homes.



- Extra reinforced bariatric crutch
- Height adjustable
- Anatomical handgrip
- Reflex on grip
- 3 models
- Disinfectable



General

This product sheet contains important information about the use of Elbow Crutch and the target group for the product. It also includes a range of specifications on Elbow Crutch.

The most recent version of this product sheet can be found on our website. Check if the version you are reading now is the most recent.

If you need a product sheet written in a larger font size, the guide is available in PDF format at cobirehab.com, which can be scaled up to the required font size.

Use

Elbow Crutch is an extra reinforced bariatric crutch designed to provide support and safety while walking for users up to 325 kg.

Elbow Crutch is used at hospitals, care centers and in private homes. Elbow Crutch should only be used on level ground and accessible terrain, indoors as well as outdoors.

Settings

The Elbow Crutch height can be adjusted by removing the clip and placing the inner tube of the cane in the desired position.

Stick the clip pin through the holes and then press the clip into place.

Long term storage

Elbow Crutch should be stored at temperatures between -10° to plus 50°C.

Cleaning and maintenance

Elbow Crutch can be cleaned with water and ordinary detergent.

The height adjustment clip pin should be regularly inspected and if necessary, replaced.

You can always contact Cobi Rehab on + 45 7025 2522 or cobi@cobi.dk if you want more information about Elbow Crutch.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Product safety notices and product recalls can be viewed on the website.

Contraindications

The user of Elbow Crutch must have physical and cognitive resources to operate the crutch in a safe manner.

Disinfection

Elbow Crutch can be disinfected with ethanol 70-85% or a chlorine product 1000-1200 ppm.

Specifications

Maximum user weight	325 kg
Height adjustable	68-110 cm
Weight	1.5 kg
Reflex on grip	Yes
Underarm support opening	14 cm
Anatomical handgrip	Yes
Material	Chrome molybdenum steel /SAE41-30
Grip	Nylon with fibre
ISO - classification	ISO 12 03 06 - 04
Expected product service life	7 years
Risk class	I
UDI-DI	5740000102889/5740000102896/5740000102902
Basic UDI	5740000100010MV
The surface temperature of the Elbow Crutch can increase or decrease when exposed to external sources of heat or cold.	

Ordering

Model	Height adjustable	Item number
Bariatric Elbow Crutch with universal handgrip	68-110 cm	0205-000-000C
Bariatric Elbow Crutch with universal handgrip, right	68-110 cm	0205-000-000RC
Bariatric Elbow Crutch with universal handgrip, left	68-110 cm	0205-000-000LC



COBI REHAB[®]
The Positioning Company



MANUFACTURER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
+45 7025 2522
cobirehab.com