



**PRODUCT SHEET**

# Rollator King

**Strong and functional rollator designed exclusively for bariatric users.**

Rollator King is intended for users weighing up to 325 kg. Rollator King can be used outdoors and indoors in institutions, care centers and in private homes. Rollator King is available in two sizes, 610 mm, and 710 mm width.



- Fits through most doorways
- Also works as chair for resting
- Available in two sizes
- Extremely strong construction
- Weight capacity up to 325 kg



## One of the strongest rollators on the market

The design of Rollator King is functional and simple.

A weight capacity up to 325 kg makes it one of the strongest rollators on the market.

Rollator King fits through doorways and folds easily for transport in motor vehicles and storage.

The Rollator King provides the user with opportunities for do activities both indoors and outdoors.

## Construction

Rollator King comes with a large, upholstered seat that provides the user with the possibility to rest and sit during activity.

The very strong and durable construction together with the solid wheels with steel forks ensures great safety and support for the user.

## Safety back rest

When the user is resting on the seat, the back rest ensures the weight of the user is evenly distributed on the seat. This way the risk of tipping is prevented significantly.

## Cleaning and disinfection

The frame, seat, handles, and safety back rest of the rollator must be cleaned with water and a regular cleaning agent.

## Long term storage

Rollator King must be stored at temperatures between -10° and 50°C.

## Transport in motor vehicle

Rollator King can easily be folded if the rollator is to be transported in a motorized vehicle.

Grasp the front edge of the seat with one hand. Hold one of the handles with the other hand.

Lift the seat and fold the rollator.

## General information

This product sheet contains important information about the use of Rollator King and the target group for the product. It also includes a range of specifications on Rollator King.

The most recent version of this product sheet can be found on our website where it also can larger font size.

You can always contact Cobi Rehab if you want more information about Rollator King.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Rollator King is equipped with stable brakes, which can lock as a parking brake or be used as a manual brake. The two types of brakes ensure control and safe maneuvering.

The rollator comes with a basket and the handles are fitted with reflectors.


It also makes the user more comfortable when sitting.

## Disinfection

The rollator's frame, seat, handles, and safety backrest can also be disinfected with ethanol 70-85% or a chlorine product 1000-1200 ppm .

The upholstered seat and the safety back rest can be exchanged when switching to a new user.

The basket and the safety back rest can be removed to take up less space.

 Rollator King must not be used as a seat in a motor vehicle.

<b>Specifications model 610 mm and 710 mm</b>	
Maximum user weight	325 kg
Total length	850 mm
Total width	740/840 mm
Width between handles	610/710 mm
Height adjustable in 2.5 cm intervals	770.5 - 990 mm
Seat width	510 mm
Seat depth	370 mm
Seat opening	610/710 mm
Seat height	500 mm
Weight	15.7 kg
Weight capacity in basket	5 kg
Foldable	Yes
Resting chair	Yes
Massive wheels	Yes
Frame	Steel/SAE4130
Seat and back	Polyurethane
Wheels	8" massive
Expected product life	7 years
ISO classification	ISO 12 06 06 - 11
Risk class	I
UDI-DI	5740000102490
Basic UDI	5740000100007N8
User manuals and product sheets are available on the website and can be scaled up to large font size for the visually impaired.	

<b>Model</b>	<b>Seat opening</b>	<b>Total width</b>	<b>Item number</b>
Rollator King, incl. basket	610 mm	740 mm	0172-050-000
Rollator King, incl. basket	710 mm	840 mm	0172-071-000