



**COBI REHAB**<sup>®</sup>  
The Positioning Company

## PRODUCT SHEET

# Bariatric Comfort Foam Mattress

**Bariatric Comfort Foam Mattress is a pressure relieving foam mattress designed for bariatric patients.**

For patients in need of pressure relief and a wide mattress. Used in hospitals, institutions, nursing homes and home care.

**Maximum user weight 325 kg.**



Medical device

Cobi Rehab and Cobi Rehab's suppliers comply with UN conventions on child and forced labor and ILO 029, C132 and C182.

## General information

This product sheet contains important information about the use of Bariatric Comfort Foam Mattress and the target group for the product.

It also includes a range of specifications on Bariatric Comfort Foam Mattress.

The most recent version of this product sheet can be found on our website where it also can larger font size.

You can always contact Cobi Rehab if you want more information about the product.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Product safety notices and product recalls can be viewed on the website.

## Use

Bariatric Comfort Foam Mattress is used in home care, nursing homes, hospitals and institutions.

Bariatric Comfort Foam Mattress is specially designed for prevention and treatment of pressure ulcers for low-risk categories.

Bariatric Comfort Foam Mattress is for users with a maximum weight of 325 Kg.

Bariatric Comfort Foam Mattress is easy to clean and to reuse.

## Contraindications

Always consult a doctor or therapist before assigning a Bariatric Comfort Foam Mattress to a patient.

## Construction

Bariatric Comfort Foam Mattress is made up of two different layers of quality foam.

The base layer is a thick base layer of Heavy Duty PU foam cut into springs.

The top layer consists of three cm viscoelastic foam, which provides comfort and pressure relief.

## Cover

The cover is removable, breathable, stretchable, and anti-bacterially treated on the surface.

The cover is protected against liquid penetration and is flame retardant. The cover can be washed, tumble dried and disinfected.

The cover has a zipper on three sides. The zipper is covered by an overlap of the cover.

## Cleaning

Bariatric Comfort Foam Mattress Cover can be cleaned with water, detergent, and a dry, clean cloth to wipe.

The cover can also be disinfected with ethanol 70-85% or a chlorine product 1000-1200ppm.

The cover can also be machine washed at 95°C.

The cover can be tumble dried at low heat or hung to dry if possible.

The cover must not be ironed.

Bariatric Comfort Foam Mattress must be thoroughly disinfected when given to a new user. This is done to avoid cross-contamination and infections.

## Storage

### Storage (and long-term storage)

Bariatric Comfort Foam Mattress must be stored lying flat and without anything else lying on top, in temperatures between 10° - 40°C.

### Maintenance

Check the Bariatric Comfort Foam Mattress regularly for damage to ensure optimal effect. If the cover is damaged, this must be replaced immediately.

## Ordering

| Model                                     | Length x width x height in cm | Item no      |
|---|-------------------------------|--------------|
| Bariatric Comfort Foam Mattress           | 200 X 100 X 18                | 0240-100-200 |
| Bariatric Comfort Foam Mattress           | 200 X 120 X 18                | 0240-120-200 |
| Bariatric Comfort Foam Mattress           | 220 X 100 X 18                | 0240-100-220 |
| Bariatric Comfort Foam Mattress           | 220 X 120 X 18                | 0240-120-220 |
| Bariatric Comfort Foam Mattress           | 220 X 140 X 18                | 0240-140-220 |
| <b>Accessories</b>                        |                               |              |
| Cover for Bariatric Comfort Foam Mattress | 200 X 100                     | 0240-100-300 |
| Cover for Bariatric Comfort Foam Mattress | 200 X 120                     | 0240-100-301 |
| Cover for Bariatric Comfort Foam Mattress | 220 X 100                     | 0240-999-010 |
| Cover for Bariatric Comfort Foam Mattress | 220 X 120                     | 0240-999-011 |
| Cover for Bariatric Comfort Foam Mattress | 220 X 140                     | 0240-999-015 |

| <b>Specifications</b>   |                                   |
|---|-----------------------------------|
| Maximum user weight   | 325 kg                            |
| Usable up to pressure ulcers  | Stage 1                           |
| ISO classification  | 18 12 18 - 25                     |
| Expected product life   | 7 years                           |
| Risk class  | I                                 |
| User manuals and product sheets are available on the website and can be scaled up to large font size for the visually impaired. |                                   |
| Environmentally sound disposal  | Return the mattress for recycling |

| <b>Mattress</b>                         |                        |
|---|------------------------|
| Length                                  | 200/220 cm             |
| Width                                   | 100/120/140 cm         |
| Height                                  | 18 cm                  |
| Weight                                  | 12/13 kg               |
| Can be applied directly to the bed base | Yes                    |
| Usable for adjustable bed base          | Yes                    |
| Mattress base material                  | Heavy Duty PU foam     |
| Mattress top material                   | Visco elastic PUR foam |
| Flame retardant mattress                | Yes                    |

| <b>Cover</b>   |                   |
|--|-------------------|
| Removable, allergy-friendly, washable and disinfectable incontinence cover | Yes               |
| Cover material   | 100% polyurethane |
| Flame retardant cover  | Yes               |
| The cover complies with the ROHS directive                                 | Yes               |



**COBI REHAB**<sup>®</sup>  
The Positioning Company

---

**MANUFACTURER**

**Cobi Rehab**  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
+45 7025 2522  
[cobirehab.com](http://cobirehab.com)