



Setup of Alternating Pressure Mattress, Max user weight 270 kg

1. Place the mattress on the bed with the hoses for the pump placed in feet part.

2. Hang the pump on the footboard and check that it is stable.

3. Connect the mattress hoses to the pump. Make sure they are connected correctly.

⚠ The hoses must not "break" or be hidden under the mattress.

4. Make sure the CPR straps are properly seated and connected the mattress.

5. Plug in the pump.

⚠ The patient cannot lie on the mattress during setup.

6. Turn on the on/off button on the right side of the pump. Wait for beep.

7. When the pump is switched on, it goes into static mode and fills the mattress up to 30 mm Hg. The pump then switches to alternating pressure.

When the pressure does not reach 30 mm Hg after 45 minutes, the alarm sound starts. Please contact your local dealer in this case.

8. Cycle time and comfort level are set as default to a mean. Use the panel to select cycle time and comfort level (PRESSURE RANGE).

9. After setup, place a loose bed sheet on the mattress.

Checking the pressure - Hand Test

Press on the air cells that are in contact with exposed areas.

The mattress must not be too hard. There must be approx. 2 - 4 cm to the bottom of the mattress.

Hand test can be done, when the user has been lying on the mattress for a minimum of 10 minutes.

Cleaning

Use water, a clean cloth, and a mild detergent. Disinfect the mattress and pump parts with ethanol 70-85% or a chlorine product of min. 1000 and max. 10,000 ppm.



QUICK GUIDE

Bariatric Alternating Pressure Mattress - 270 kg

The pump monitor measures constantly the pressure in the mattress. At too low pressure, the pump automatically fills the mattress to the set level.

ON/OFF - No 1

The power switch is on the right side of the pump.

FUSE - No 2

The pump has two fuses.

CORD - No 3

The cord is fixed on the pump.

PRESSURE RANGE - No 4

The mattress has 8 levels of comfort, increasing with 34 Kg per level.

CYCLE TIME - No 5

There are 4 cycle times, 10, 15, 20 or 25 min. 10 min = 2 diodes. 15 min = 4 diodes etc.

ALARM RESET - No 6

The alarm sound is on, and the ALARM RESET light flashes on power failure or pressure loss.

Press the button to mute or/and unmute the sound of the alarm.

STATIC - No 7

Static function stops alternating pressure function.

ALTERNATING - No 7

Press ALTERNATING to switch to alternating pressure function.

AUTOFIRM - No 8

Autofirm is for care situations and fills up the mattress to maximum 70 mm Hg. Continues for 30 min. Press AUTOFIRM to stop the function before the 30 min. The pump switches automatically back to alternating pressure.

