



Cobi Sky Positioning Mattress

- for the difficult positionings

Cobi Sky Positioning Mattress is designed to ensure ample pressure distribution, stability, comfort, and full support.

Cobi Sky Positioning Mattress is used in care centers, residences and in home care for users with the need for deep and pressure-relieving positioning. Recommended for sensitive users with neurological or pain symptoms.

User weight from 30-110 kg.



- Modellable mattress for micro positioning.
- Easy to install, no pump.
- Pressure ulcer prevention and relief.
- Foam frame ensuring stability.
- Stretchable and breathable incontinence cover.
- Perforated middle layer ensuring optimal ventilation.



Cobi Sky Positioning Mattress

Mattress

Cobi Sky Positioning Mattress is a complete mattress with several layers. The top layer consists of a cover with small Krøyer beads. The inner cover is sewn in channels so the contents of each individual channel can be moved around. The mattress can be shaped exactly to the user's needs ensuring optimal support and pressure relief, while at the same time reducing the risk of shear.

Under the layer with channels and Krøyer beads, there is a springy middle layer with perforations ensuring optimal ventilation of the mattress.


A foam frame around and under the two layers ensures stability when the user is sitting on the edge of the bed.

Application

Push the filling around

Cobi Sky Positioning Mattress meets the challenges of pressure sores without compromising the stability.

This is partly due to the mattress channels that can be pushed around. In this way, the mattress can be shaped exactly to the needs of the user.

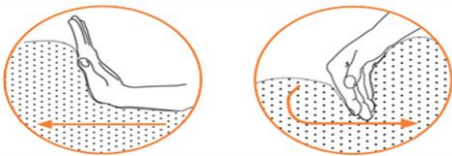


Positioning with indentations

In the case of existing bedsores, a hole can be made for relief by pushing the beads filling around.

Make a hole with your hand below the point where the pressure is to be relieved.

The hole can subsequently be smoothed out again when there is no longer need for relief.

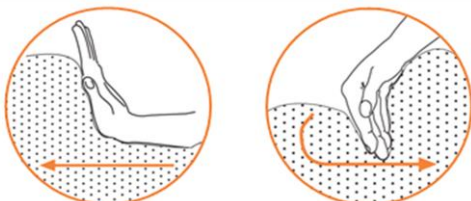

Cover

The grey outer cover of the foam mattress is an incontinence cover which is stretchable, breathable, and washable. The grey cover covers the entire mattress and closes with a zip.

The outer cover sits loosely, which allows nursing staff to push and shape the filling in the mattress without the tightness of the cover cancelling out the positioning and function of the filling.

The white inner cover is an ultra-thin, flexible, and stretchable Lycra cover.

Specifications

Cobi Sky mattress incl. grey outer cover: 2000 mm (L) x 850 mm (W) x 150 mm (H). Item number 2100-085-200.

Micro positioning

Small adjustments of the mattress are made continuously, for instance, for micro positioning. This can be done to create small pressure-distributing changes in the mattress without the user having to be actively involved, for example, during night inspections, where the user's sleep should not be disturbed.

This is also done if the user cannot handle significant turns, lifting or moving in the bed.

Cleaning and maintenance

Daily cleaning of the grey outer cover can be done with water, detergent, and a dry, clean cloth for wiping.

An alternative to cleaning with water is disinfection with ethanol 70-85% or a chlorine product 1000-1200 ppm.

The grey outer cover can be washed in a washing machine at 60°C and dried in a tumble dryer on low heat or hung to dry if possible. The cover must not be ironed.

The white inner cover has a service life of 8 washes. It is washed in a laundry bag in a washing machine at 60°C. The inner cover must not be dried in a tumble dryer but must be hung up.

Transport

Cobi Sky is equipped with four handles. Two on each side. Grab the handles when moving the mattress.

Cobi Sky can be rolled up for shorter periods. A roll-up strap is included.