



PRODUCT SHEET

Cobi Sky Positioning Mattress

- for the difficult positionings

Cobi Sky Positioning Mattress is designed to ensure ample pressure distribution, stability, comfort, and full support.

Cobi Sky Positioning Mattress is used in care centers, residences and in home care for debilitated users with, among other things, neurological symptoms, severe pain, or unstable fractures with the need for deep and pressure-relieving positioning.

User weight from 30-110 kg.



- Modellable mattress for micro positioning.
- Easy to install, no pump.
- Pressure ulcer prevention and relief.
- Foam frame ensures stability.
- Stretchable and breathable incontinence cover.
- Perforated middle layer that ensures optimal ventilation.



Cobi Sky Positioning Mattress

Mattress

Cobi Sky Positioning Mattress is a complete mattress with several layers. The top layer (below the white and grey covers in the image) consists of a cover with small Krøyer beads. The cover is sewn in channels, which means that the contents of each individual channel can be moved around. This means that the mattress can be shaped exactly to the position that the user needs.

In other words, the user sinks into the mattress, which ensures optimal support and pressure relief, while at the same time reducing the risk of shear.


Under the layer with channels and Krøyer beads, there is a springy middle layer with perforations that ensures optimal ventilation of the mattress.

Application

Push the filling around

Cobi Sky Positioning Mattress meets the challenges of pressure sores without compromising the stability.

This is partly due to the construction of the mattress with channels that can be pushed around. In this way, the mattress can be shaped exactly to the needs of the user.

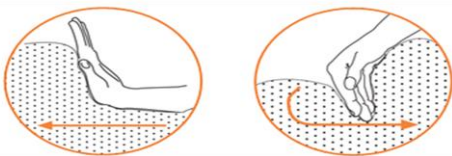


Positioning with indentations

In the case of existing bedsores, a hole can be made for relief by pushing the beads filling around.

Make a hole with your hand below the point where the pressure is to be relieved.

The hole can subsequently be smoothed out again when there is no longer need for relief of, for example, the heel.

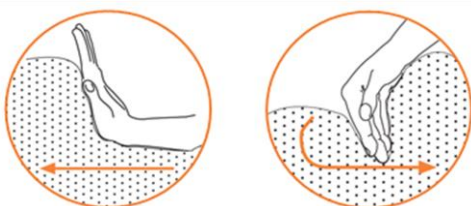

A foam frame around and under the two layers ensures stability when the user is sitting on the edge of the bed.

Cover

The grey outer cover for the foam mattress is an incontinence cover which is stretchable, breathable, and washable. The grey cover covers the entire mattress and closes with a zip.

The outer cover sits loosely, which allows nursing staff to push and shape the filling in the mattress without the tightness of the cover cancelling out the positioning and function of the filling.

The white inner cover is an ultra-thin, flexible, and stretchable Lycra cover.

Micro positioning

Small adjustments of the mattress are made continuously, for instance, for micro positioning. This is done, among other things, to create small pressure-distributing changes in the mattress without the user having to be actively involved, for example, during night inspections, where the user's sleep should not be disturbed.

This is also done if the user cannot handle significant turns or lifting and moving in the bed.

Cleaning and maintenance

Daily cleaning of the outer cover can be done with water, detergent, and a dry, clean cloth for wiping.

An alternative to cleaning with water is disinfection with ethanol 70-85% or a chlorine product 1000-1200 ppm.

The grey outer cover can be washed in a washing machine at 60°C and dried in a tumble dryer on low heat or hung to dry if possible. The cover must not be ironed.

The white inner cover has a service life of 8 washes. The inner cover is washed in a washing bag in a washing machine at 60°C. The inner cover must not be dried in a tumble dryer but must be hung up.

Transport

Cobi Sky is fitted with four handles. Two on each side. Hold the handles when moving the mattress.

Cobi Sky can be rolled up for shorter periods. A roll-up strap is included.

Specifications	
Max. user weight	30-110 kg
Intended for pressure sore category	1-4
Length	2000 mm
Width	850 mm
Height	150 mm
Weight of mattress	13 kg
Can be used directly on the bed base	Yes
Can be used directly on the adjustable bed base	Yes
Carrying handles	Four
Mattress	Polyamide fabric, expanded polystyrene beads and polyester fibres
Foam mattress frame	Polyurethane foam, polyester fibres
Fire-retardant mattress	Yes
Fire-retardant outer cover	Yes
Incontinence cover	Yes
Hypoallergenic cover	Yes
Washable and disinfectable cover	Yes
Grey outer cover	44% polyamide, 56% polyurethane
ISO classification	04 33 06-05
Expected product service life	5 years
Eco-friendly disposal	Return the mattress for recycling

Model	Length	Width	Height	Item number
Cobi Sky mattress incl. grey outer cover	2000 mm	850 mm	150 mm	2100-085-200
Spare parts				
Grey top and bottom cover (zipped together)				2100-999-010
Foam mattress with a fibre layer				Contact Cobi Rehab
Krøyer beads mattress part				on +45 7025 2522
White inner cover				when purchasing
Detachable male-part clasp				spare parts.



COBI REHAB[®]
The Positioning Company



**MANUFACTURER AND
DEALER**

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
+45 7025 2522
cobirehab.com