



## PRODUCT SHEET

# Lasal Flotation Cushions

### Everyone enjoys a good experience in the pool

Lasal Flotation Cushions are used for positioning in the pool during training and water therapy. The Flotation Cushions are developed especially for disabled children and adults but can also be used by people in need of extra support in water.

Lasal Flotation Cushions can be used in pools with chlorinated water.



- Provides great float support.
- Ensures stability and security, which provides increased mobility and more opportunities for training.
- Helps therapist to be able to perform relevant and beneficial exercises with user.
- Many different shapes and sizes of cushions.



## Use

Lasal Flotation Cushions are developed especially for disabled children and adults but can also be used by people in need of extra support in the water.

The cushions are used for positioning in the pool during training as well as during various forms of water therapy. The cushions contribute to more training options and ensure a comfortable water experience for the user.

Lasal Flotation Cushions can be used in chlorinated water.

## Products

The Flotation Cushions are made in several models and sizes for both children and adults. The different models can be used individually or combined according to each user's needs.

The cushions have a great buoyancy. When the cushions are filled with water, they ensure stability and support, which means a safe and comfortable experience for the user in the water.

The cushions can be placed around the user before the user is helped into the water.

Lasal Flotation Cushions, despite their buoyancy, must not be considered as life jackets or other rescue equipment.

The cushions have a blue and a black side. It is recommended that the black side is to be turned towards the body.

### Lasal Flotation Cushions comes in different shapes and sizes:

- Collar Float: 4 sizes
- Neck Floating Collar: 2 sizes
- Banana shaped Flotation Cushion: 3 sizes
- U- Cushion: 1 size
- T-shaped Flotation Cushion: 2 sizes
- Flower Float: 2 sizes
- Float Stabilizer: 2 sizes

## Cleaning and Maintenance

Rinse the outside of the cushions clean of chlorine with plenty of non-chlorinated water after use.

Hang the cushions for drying. The blue net side is turned down so the water can flow freely out of the cushion. The drying time is long!

The cushions can be hand washed with soap and water or sprayed with a disinfectant.

Please note that the pH of the disinfectant used should not exceed 11.4.

Use ordinary detergent, but no fabric softener!



Collar Float, T-shaped and Banana shaped Flotation Cushions.



T-shaped and Banana shaped Flotation Cushions.



Flower Float

If necessary, the cushions can be washed at 60°C in a washing machine. Due to their size and weight, the cushions must be washed in a washing machine large enough to handle the size of them.

Make sure to use a laundry bag for each cushion. The laundry bag must have fine mesh in order to protect the washing machine from the cushion filling. If the filling comes out of the cushion, the washing machine can be damaged.

Use of a drying cabinet is of max. 60°C. Higher temperature destroys the filling as the polystyrene beads melt above 71°C.


# Lasal Flotation Cushions - overview

Sizes specified in length x width x neck hole



T-shaped Flotation Cushion, S  
Size: 63x39 cm  
Item no: 3002-9027-000

T-shaped Flotation Cushion, L  
Size: 81x54 cm  
Item no: 3002-9017-000

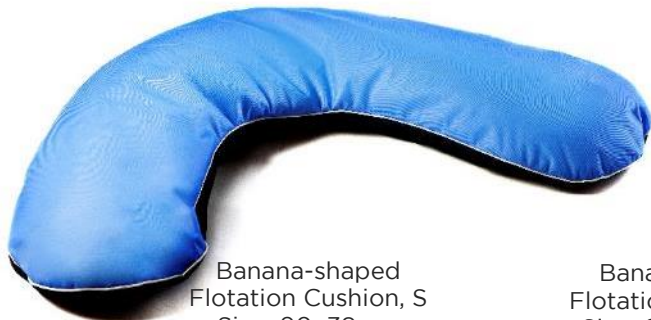

Banana-shaped Flotation Cushion, S  
Size: 90x38 cm  
Item no: 3002-9811-000

Banana-shaped Flotation Cushion, M  
Size: 98x38-33 cm  
Item no: 3002-9111-000



U-Cushion  
Size: 55x19 cm  
Item no: 3002-9911-000

Banana-shaped Flotation Cushion, L  
Size: 125x30-37 cm  
Item no: 3002-9101-00



Collar float, XS  
Size: 50x34x23 cm  
Item no: 3002-9421-000

Collar float, S  
Size: 59x39x25 cm  
Item no: 3002-9415-000

Collar float, M  
Size: 63x42x30 cm  
Item no: 3002-9411-000

Collar float, L  
Size: 68x46x35 cm  
Item no: 3002-9401-000



Neck Collar, S  
Size: 41x41x26 cm  
Item no: 3002-9321-000

Neck Collar, L  
Size: 51x51x27 cm  
Item no: 3002-9311-000



Bathing Flower, S  
Size: 68x20 cm  
Item no: 3002-9600-000

Bathing Flower, L  
Size: 77x28 cm  
Item no: 3002-9610-000



Float Stabilizer, S  
Size: 60-37x18-25 cm  
Item no: 3002-9620-000

Float Stabilizer, L  
Size: 93-55x24-29 cm  
Item no: 3002-9630-000

## Specifications

Filling	Fire-retardant EPS beads (expanded polystyrene)
Cover- black side	100% polyester, not flame retardant
Cover- blue side	100% polyester, not flame retardant
<b>Cleaning</b>	
<p>After use, rinse the cushion clean for chlorine with non-chlorine-containing water. Hang them up and let them drip off.</p> <p>The cushions can be hand washed with soap and water or sprayed with a disinfectant.</p> <p>Note that the pH of the applied disinfectant should not be higher than 11.4.</p> <p>Flotation Cushions can be machine washed at 60°C. Due to its size and weight, Flotation Cushions must be washed in a washing machine large enough to handle the size of the cushions.</p> <p>Make sure to use a laundry bag with one cushion in each bag. The laundry bag must have fine mesh in order to protect the washing machine from the cushion filling. If the filling comes out of the cushion, the washing machine can be damaged.</p> <p>Use ordinary detergent, but no fabric softener!</p> <p>Drying in a drying cabinet must not exceed 60°C. Temperatures above 70°C destroy the filling as the polystyrene beads melt above 71°C.</p>	
Storage	Do not expose the cushions to high temperatures, prolonged and intense sunlight, or other radiation.
ISO classification	09 33 39 01
Expected lifetime	2 years
UDI-DI	5740000100022N4

---

Please contact Cobi Rehab for questions

**cobi@cobi.dk** or +45 **7025 2522**.

---

**Cobi Rehab**  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
+45 7025 2522  
cobirehab.com

## Order

Model	Length cm	Width cm	Circumference of neck hole	Item number
T-shaped Flotation Cushion S	63	39		3002-9027-000
T-shaped Flotation Cushion L	81	54		3002-9017-000
Banana-shaped Flotation Cushion S	90	38		3002-9811-000
Banana-shaped Flotation Cushion M	98	38/33		3002-9111-000
Banana-shaped Flotation Cushion L	125	30/37		3002-9101-000
Neck Collar S	41	41	26	3002-9321-000
Neck Collar L	51	51	27	3002-9311-000
Collar Float XS	50	34	23	3002-9421-000
Collar Float S	59	39	25	3002-9415-000
Collar Float M	63	42	30	3002-9411-000
Collar Float L	68	46	35	3002-9401-000
U-Cushion	55	19		3002-9911-000
Flower Float S	68		20	3002-9600-000
Flower Float L	77		28	3002-9610-000
Float Stabilizer, S	60/37	18/25		3002-9620-000
Float Stabilizer, L	93/55	24/29		3002-9630-000



**COBI REHAB<sup>®</sup>**  
The Positioning Company