



Flotation and Hygiene Cushions

Washing and Drying

The cushions can be washed by hand. By hand wash, the cushions can also be disinfected, e.g. by following local guidelines.

The cushions can with precaution be washed in a large washing machine at maximum 60° C.

IMPORTANT !!! The cushions **MUST** be washed in an intact machine wash bag. The machine wash bag must have fine mesh, to protect the washing machine from the cushion filling.

If the filling comes out of a cushion, the washing machine can be damaged!!!

IMPORTANT !!! The washing machine must be large enough to handle the size of the cushions.

Always use intact machine wash bags, when washing the Flotation and Hygiene cushions.

The cushions can be dried in drying cabinet at medium heat (max 60°C). The cushions can with precaution also be dried in a tumble dryer at medium heat (max 60°C).

In tumble dryer it is **IMPORTANT to use an intact machine wash bag!!!** The machine wash bag must have fine mesh, to protect the tumble dryer from cushion filling!

If the filling comes out of a cushion, the tumble dryer can be damaged!!!

After **use in chlorinated pool water** rinse the cushion thoroughly in non-chlorinated water. When drying the flotation cushion must be placed with the blue mesh facing downwards, allowing the water to drain from the cushions.

After use in hygiene situations the cushion can be disinfected with a detergent with a pH below 11,4. E.g. with laundry disinfectants to kill bacteria and e.g. Coronavirus. ECA water can also be used (tested with 50 ppm).