



PRODUCT SHEET

Bariatric Comfort Foam Mattress

Bariatric Comfort Foam Mattress is a pressure relieving foam mattress designed for bariatric patients.

For patients in need of pressure relief and a wide mattress. Used in hospitals, institutions, nursing homes and home care.

Maximum user weight 325 kg.



Medical device

Cobi Rehab and Cobi Rehab's suppliers comply with UN conventions on child and forced labor and ILO 029, C132 and C182.

General information

This product sheet contains important information about the use of Bariatric Comfort Foam Mattress and the target group for the product.

It also includes a range of specifications on Bariatric Comfort Foam Mattress.

The most recent version of this product sheet can be found on our website where it also can larger font size.

You can always contact Cobi Rehab if you want more information about the product.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Product safety notices and product recalls can be viewed on the website.

Use

Bariatric Comfort Foam Mattress is used in home care, nursing homes, hospitals and institutions.

Bariatric Comfort Foam Mattress is specially designed for prevention and treatment of pressure ulcers for low-risk categories.

Bariatric Comfort Foam Mattress is for users with a maximum weight of 325 Kg.

Bariatric Comfort Foam Mattress is easy to clean and to reuse.

Contraindications

Always consult a doctor or therapist before assigning a Bariatric Comfort Foam Mattress to a patient.

Construction

Bariatric Comfort Foam Mattress is made up of two different layers of quality foam.

The base layer is a thick base layer of Heavy Duty PU foam cut into springs.

The top layer consists of three cm viscoelastic foam, which provides comfort and pressure relief.

Cover

The cover is removable, breathable, stretchable, and anti-bacterially treated on the surface.

The cover is protected against liquid penetration and is flame retardant. The cover can be washed, tumble dried and disinfected.

The cover has a zipper on three sides. The zipper is covered by an overlap of the cover.

Cleaning

Bariatric Comfort Foam Mattress Cover can be cleaned with water, detergent, and a dry, clean cloth to wipe.

The cover can also be disinfected with ethanol 70-85% or a chlorine product 1000-1200ppm.

The cover can also be machine washed at 95°C.

The cover can be tumble dried at low heat or hung to dry if possible.

The cover must not be ironed.

Bariatric Comfort Foam Mattress must be thoroughly disinfected when given to a new user. This is done to avoid cross-contamination and infections.

Storage

Storage (and long-term storage)

Bariatric Comfort Foam Mattress must be stored lying flat and without anything else lying on top, in temperatures between 10° - 40°C.

Maintenance

Check the Bariatric Comfort Foam Mattress regularly for damage to ensure optimal effect. If the cover is damaged, this must be replaced immediately.

Ordering

Model	Length x width x height in cm	Item no
Bariatric Comfort Foam Mattress	200 X 100 X 18	0240-100-200
Bariatric Comfort Foam Mattress	200 X 120 X 18	0240-120-200
Bariatric Comfort Foam Mattress	220 X 100 X 18	0240-100-220
Bariatric Comfort Foam Mattress	220 X 120 X 18	0240-120-220
Bariatric Comfort Foam Mattress	220 X 140 X 18	0240-140-220
Accessories		
Cover for Bariatric Comfort Foam Mattress	200 X 100	0240-999-009
Cover for Bariatric Comfort Foam Mattress	200 X 120	0240-999-011
Cover for Bariatric Comfort Foam Mattress	220 X 100	0240-999-010
Cover for Bariatric Comfort Foam Mattress	220 X 120	0240-999-012
Cover for Bariatric Comfort Foam Mattress	220 X 140	0240-999-015

Specifications	
Maximum user weight	325 kg
Usable up to pressure ulcers	Stage 1
ISO classification	18 12 18 - 25
Expected product life	7 years
Risk class	I
User manuals and product sheets are available on the website and can be scaled up to large font size for the visually impaired.	
Environmentally sound disposal	Return the mattress for recycling

Mattress	
Length	200/220 cm
Width	100/120/140 cm
Height	18 cm
Weight	12/13 kg
Can be applied directly to the bed base	Yes
Usable for adjustable bed base	Yes
Mattress base material	Heavy Duty PU foam
Mattress top material	Visco elastic PUR foam
Flame retardant mattress	Yes

Cover	
Removable, allergy-friendly, washable and disinfectable incontinence cover	Yes
Cover material	100% polyurethane
Flame retardant cover	Yes
The cover complies with the ROHS directive	Yes



COBI REHAB[®]
The Positioning Company

MANUFACTURER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
+45 7025 2522
cobirehab.com