

PRODUCT SHEET Lasal Dental Cushions

Feel comfortable at the dentist.

Dental Cushions are designed to ensure a comfortable base in the dental chair during treatment for patients with different needs and challenges.

For use in dental clinics.



- Easy to adjust to patient and chair.
- Washable covers in different materials.
- Cushions for support of the head, neck, arms, legs, upper back, and lower back.
- Developed in cooperation with The Mun-H Centre in Odontology in Gothenburg, Sweden.
- Can minimize involuntary movements and tensity for the patient.
- 12 cushions in different shapes and sizes.



Cobi Rehab and Cobi Rehab suppliers comply with the UN conventions on child labour and forced labour as well as ILO 029, C132 and C182.

Use

For patients with physical and mental challenges being seated or put down in a dental chair can cause discomfort, anxiety, or restlessness. That is why Cobi Rehab has developed a dental cushion series.

Lasal Dental Cushions can be used alone or in combination to maximize the feeling of support, comfort and safety when visiting the dentist.

Dental fear can also be a reason for using Lasal Dental Cushion.

Regardless of the underlying reason, the aim is to avoid an uncomfortable position that may affect the muscle tension in an undesired way and thus complicating dental treatment.

Construction Dental Cushions

Lasal Dental Cushions are available in multiple shapes and sizes. The combination of small beads of polystyrene in a smooth inner cushion contributes to each cushion adjusting to the body shape as well as the dental chair.

Lasal Dental Cushions are mouldable and therefore more stable than ordinary cushions. There are four cushions supporting the area around the head, one cushion in two sizes supporting arms and four cushions supporting e.g., legs and lower legs.

All cushions can be purchased with removable and washable hygiene or jersey covers. To prolong the durability of the cushions, we always recommend using covers on the cushions.

Which cover to choose?

For the working area around head and upper body we recommend dental cushions with hygiene cover which are washable at high temperatures and easy to disinfect.

Cushions for seat, legs and feet can be used with both covers. The advantage of a jersey cover in this area is that the surface of the cover ensures that it does not slip on the surface of the dental chair.

Cleaning and maintenance

Wipe the cushions with a clean cloth wrung out in water with a mild detergent. The cushions must be completely dry before they are used again or put away.

Desinfection and wash

For surface disinfection of the cushions use disposable cloths with ethanol (e.g., Wet Wipe 70 %) or a chlorine product 1000-1200 ppm (e.g., Wet Wipe Chlorine Desinfection).



U-Shaped Neck Cushions with hygiene cover.



Safe Guarder (adult) with hygiene cover, Leg Relaxer (adult) with hygiene cover, oblong with jersey cover.




Safe Guarder (child) with hygiene cover, Crescent Cushion with hygiene cover, Seat Cushion (child) with jersey cover.

The hygiene cover is washable at 95° C and can be tumble dried. The jersey is washable at 60° C and can be tumble dried at low temperatures. Avoid rinse products with solvent.

Lasal Dental Cushions without cover do not withstand machine washing but can be cleaned with a mild detergent or disinfected.

Lasal Dental Cushions

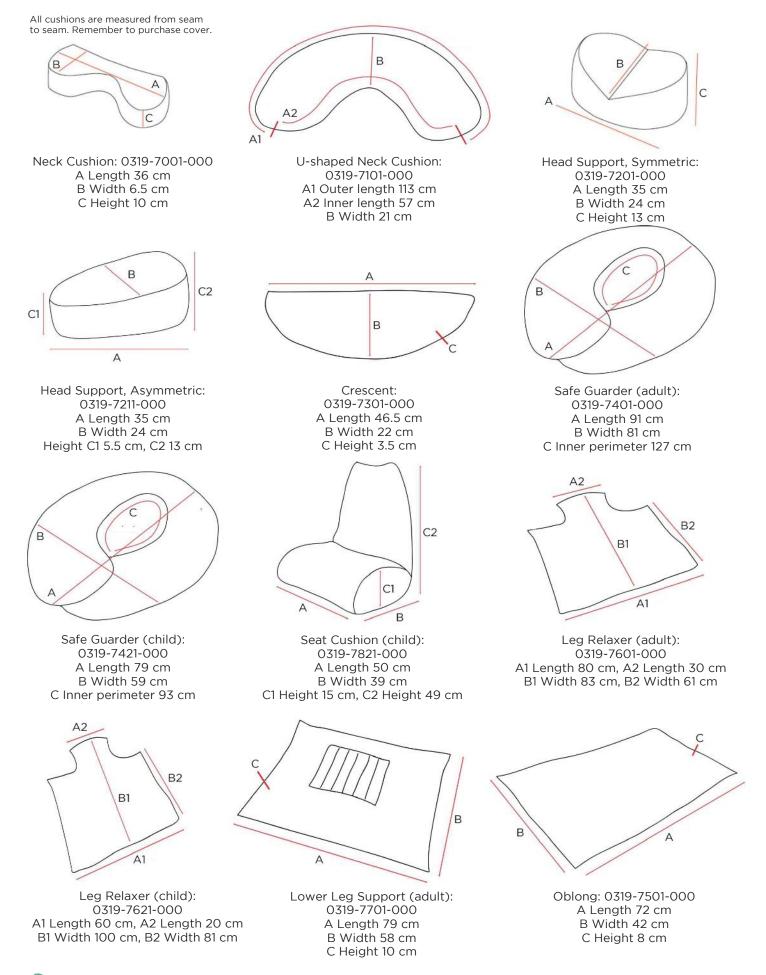

Specifications

Filling	EPS beads (Expanded Polystyrene)		
Inner cover	PU-coated flame retardant polyester textile. Mold and fungus resistant.		
Hygiene cover	PU-coated flame retardant polyester textile. Mold and fungus resistant.		
Jersey cover	Dermofresh. Flame retardant, Tencel /Lenzing , OEKO-TEX Standard 100, Class I, washable, PU coated on backside.		
	Textile surface 100% natural origin. Halogen-free and non-toxic product.		
Cleaning	The cushion is not washable, but can be disinfected with e.g., Wet Wipes or cleaned with a mild detergent.		
Washable covers	Yes		
Storage	Do not expose the cushions to high temperatures, prolonged and intense sunlight, or other radiation.		
ISO Code Danish	AssistData 09.07.06		
Expected Product Service Life			
UDI-DI	5740000100014N5		

You can always contact Cobi Rehab on +45 7025 2522 or cobi@cobi.dk for more information about Lasal Dental Cushions.

Cobi Rehab | Fuglebækvej 1D, DK-2770 Kastrup, Denmark | cobi@cobi.dk | cobirehab.com

Order

Model	Length cm	Width cm	Height cm	Color	Item number
Neck Cushion	36	6.5	10		0319-7001-000
Hygiene cover				Grey	0319-7001-010
U-Shaped Neck Cushion	57/113	21			0319-7101-000
Hygiene cover				Grey	0319-7101-010
Head Support, Asymmetric	35	24	5.5/13		0319-7201-000
Hygiene cover				Grey	0319-7201-010
Head Support, Symmetric	35	24	13		0319-7211-000
Hygiene cover				Grey	0319-7211-010
Crescent	46.5	22	3.5		0319-7301-000
Hygiene cover				Grey	0319-7301-010
Safe Guarder, adult	91	81	127 (inner perimeter)		0319-7401-010
Hygiene cover				Grey	0319-7401-010
Safe Guarder, child	79	59	93 (inner perimeter)		0319-7421-000
Hygiene cover				Grey	0319-7421-010
Oblong	72	42	8		0319-7501-000
Hygiene cover				Grey	0319-7501-010
Jersey Cover				Grey	0319-7501-015
Leg Relaxer, adult	30/80	61/83			0319-7601-015
Hygiene cover				Grey	0319-7601-010
Jersey Cover				Grey	0319-7601-015
Leg Relaxer, child	20/60	81/100			0319-7621-000
Hygiene cover				Grey	0319-7621-010
Jersey Cover				Grey	0319-7612-015
Support Band (Leg Relaxer)					0319-7606-000
Lower Leg Support, adult	79	58	10		0319-7701-000
Hygiene cover				Grey	0319-7701-010
Jersey Cover				Grey	0319-7701-015
Seat Cushion, child	50	39	15/49		0319-7821-000
Hygiene cover				Grey	0319-7821-010
Jersey Cover				Grey	0319-7821-015





MANUFACTEURER

Cobi Rehab Fuglebækvej 1D DK-2770 Kastrup Denmark cobi@cobi.dk

cobirehab.com