



PRODUCT SHEET

Bariatric Visco Comfort Wheelchair Cushion

A standard wheelchair cushion providing pressure relief for bariatric users.

Visco Comfort Wheelchair Cushion is intended for users weighing up to 325 kg.



- Anatomically shaped
- Comes in several sizes
- Different top and base layer
- Distributes pressure to a larger surface
- Weight capacity up to 325 kg



General information

This product sheet contains important information about the use of Bariatric Visco Comfort Wheelchair Cushion and the target group for the product. It also includes a range of specifications on Bariatric Visco Comfort Wheelchair Cushion.

The most recent version of this product sheet can be found on our website. Check if the version you are reading now is the most recent.

If you need a product sheet written in a larger font size, the guide is available in PDF format at cobirehab.com, which can be scaled up to the required font size.

Use

Bariatric Visco Comfort Wheelchair Cushion is developed for bariatric users.

Visco Comfort Wheelchair Cushion is used in wheelchairs in hospitals, in nursing homes, and private homes.

Visco Comfort Wheelchair Cushion is an anatomically shaped wheelchair cushion. It consists of a moulded 110 mm base made of a Heavy Duty PUR foam and with a 30 mm visco-elastic foam on the top.

The moulded base provides increased stability and seat support.

Long term storage

Visco Comfort Wheelchair Cushion must be stored dark at temperatures between -10° and 50° C.

Cleaning and maintenance

Visco Comfort Wheelchair Cushion cannot be washed in a washing machine.

The incontinence cover can be cleaned with water and a regular detergent. The cover can also be machine washed at max 95°C and tumble dried.

You can always contact Cobi Rehab on + 45 7025 2522 or cobi@cobi.dk if you want more information about Bariatric Visco Comfort Wheelchair Cushion.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Product safety notices and product recalls can be viewed on the website.

Visco Comfort Wheelchair Cushion has a two way stretch vapour permeable incontinence and flame resistant cover.

The included incontinence cover is washable, breathable, antibacterial treated and fire retardant.

Settings

Visco Comfort Wheelchair Cushion is placed in a wheelchair with the zipper facing downwards.

Contraindications

There are no contraindications to the use of the Visco Comfort Wheelchair Cushion.

Disinfection

Visco Comfort Wheelchair Cushion can also be disinfected with e.g., Wet Wipe 70% or a chlorine product 1000-1200 ppm (e.g., Wet Wipe Chlorine Desinfection).

Specifications

Maximum user weight	325 kg
Weight	3 kg
Seat width	560/610/660/710/810/910/960/1000 mm
Seat depth	500 mm
Seat height	140 mm
Base cushion	HD PUR foam and visco elastic PUR foam
Incontinence cover	Yes
ISO classification	18 10 06 01
Expected product service life	3 years
Risk class	I
Basic UDI	5740000100011MX
The surface temperature of the Visco Comfort Wheelchair Cushion can increase or decrease when exposed to external sources of heat or cold.	

Ordering

Model (+ depth + height in mm)	Width in mm	Item number
Visco Comfort Cushion with incontinence cover (500 mm + 140 mm)	560	0233-056-050
Visco Comfort Cushion with incontinence cover (500 mm + 140 mm)	610	0233-061-050
Visco Comfort Cushion with incontinence cover (500 mm + 140 mm)	660	0233-066-050
Visco Comfort Cushion with incontinence cover (500 mm + 140 mm)	710	0233-071-050
Visco Comfort Cushion with incontinence cover (500 mm + 140 mm)	810	0230-081-050
Visco Comfort Cushion with incontinence cover (500 mm + 140 mm)	910	0233-091-050
Visco Comfort Cushion with incontinence cover (500 mm + 140 mm)	960	0233-096-050
Visco Comfort Cushion with incontinence cover (500 mm + 140 mm)	1000	0233-100-050



COBI REHAB



MANUFACTURER

Cobi Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
+45 7025 2522
cobirehab.com