



## Product sheet

### Kamille Back Support

#### Unique back support suitable for asymmetrical adjustments.

Asymmetrical adjustments make it easier to address anatomical challenges such as scoliosis, increased lateral flexion in the truncus, or asymmetrical tissue, for example, due to overweight, hernias, or edema.

The Kamille Back Support ensures comfort, stability, and functionality for wheelchair users in need of a strong back support. Designed entirely by Cobi Rehab's own engineers and therapists. User weight up to 140 kg.



- Asymmetric adjustments of the backrest
- Adjustable side supports
- Positioning of the back can be adjusted in height and depth
- High recyclability, as it can be readjusted for a new user
- Safe and stable positioning solution
- Can be mounted with or without body supports
- Compatible with most standard wheelchairs
- Can be equipped with various types of headrests.



## General

This product sheet contains important information on the Kamille Back Support Wheelchair's use and target group. It also includes a range of specifications for the Kamille Back Support.

The latest version of this product sheet can be found on our website. Check that you are using the latest version.

If you need a version printed in a larger font size, it is available in PDF format at [cobirehab.com](http://cobirehab.com), where it can be scaled up to the desired font size.

## Kamille Back Support

The Kamille Back Support can be adjusted asymmetrically. This makes it easier to address anatomical challenges such as scoliosis, increased lateral flexion of the truncus or asymmetric tissue caused by e.g., overweight, hernias or oedema.

Asymmetrical adjustments are possible thanks to the three aluminium plates that make up the back system. Two of the plates can be moved independently of each other, even though all three plates are connected.

The back system consists of the back base, which is mounted in six slots. The slots allow the back base to be adjusted in height.

The side plates, mounted in two slots, can be loosened using ordinary tools. When the two nuts are loosened, it is possible to adjust the width and angle of the back.

## Cleaning and maintenance

The cushions can be wiped down with a damp cloth and mild soapy water.

### Disinfection

Back and cushions can also be disinfected with e.g., a Wet Wipe 70% or a chlorine product of 1,000-1,200 ppm.

## Storage and transport

When the Kamille Back Support is not in use, it should be stored in dry conditions at a temperature between 0° and +45°C and a humidity level below 75%.

The above recommendations for storage also apply for storage periods exceeding 4 months.

Please call us on +45 7025 2522 or send an e-mail to [cobi@cobi.dk](mailto:cobi@cobi.dk) for more information about the Kamille Back Support.

Cobi Rehab reserves the right to change the product specifications without prior notice.

Product safety notices and product recalls can be viewed on the website.

### Padded body supports (additional)

Specially developed body supports placed in slots on the back system contribute to near-body truncus stability if the user has e.g., scoliosis or decreased/increased thoracic tone.

### Cushion system

The Kamille Back Support is equipped with a complete and unique cushion system that ensures optimal pressure distribution and near-body positioning.

The cushions are filled with a mix of polystyrene beads and viscoelastic foam, which together contribute to increased sensory stimuli. Each cushion has its own zipper.

### Cover

For each size of the Kamille Back Support, there is a stretchable, breathable, and washable jersey cover with PU coating on the inside. The cover is placed over the back to avoid soiling.

All cushions can be changed when transferring to a new user.

The jersey cover with PU coating on the inside can be washed at 60°C and tumble-dried.

### Transport

The Kamille Back Support alone is suitable for land and/or air transport. When the Kamille Back Support is attached to a wheelchair, the suitability for transport depends on the specific wheelchair base in question.

## Specifikationer

User weight	According to the weight of the wheelchair used
Material	Aluminium
Height	500 mm + 20,5 mm
Width	340 - 520 mm
Weight	App. 2.3 - 2.6 kg
Cover (additional purchase)	Yes
Cushion surface	Stretchable PU padding
Cushions filling	Viscoelastic "foam-worms" and EPS beads
Compatible with headrest and comfort arms (additional purchase)	Yes
Compatible with the Kamille padded body supports (additional purchase)	Yes
Reusable	Yes
Disinfectable	Yes
Expected product lifespan	7 years
User manuals and product sheets are available on the website and can be scaled up to a large font size for visual impairment.	



## Ordering and accessories

Model	Width mm	Item number
Kamille Comfort Back Support, S	340 - 400	0350-999-850
Kamille Comfort Back Support, M	400 - 460	0350-999-852
Kamille Comfort Back Support, L	460 - 520	0350-999-854
<b>Accessories</b>		
Cover for Back Support, str. S	-	0350-999-555
Cover for Back Support, str. M	-	0350-999-556
Cover for Back Support, str. L	-	0350-999-557
Comfort arms with weight, R	-	0350-999-430
Comfort arms with weight, L	-	0350-999-431
Comfort arms without weight, R	-	0350-999-432
Comfort arms without weight, L	-	0350-999-433
Comfort arms stretch cover, R	-	0350-999-575H
Comfort arms stretch cover, L	-	0350-999-575V
Padded body support, R	-	0350-999-216H
Padded body support, L	-	0350-999-216V
Padded body support, Small R	-	0350-999-217H
Padded body support, Small L	-	0350-999-217V
Extra padded cover for padded body support	-	0350-999-487
Extra padded cover for padded body support, Small	-	0350-999-488
Headrest	-	0350-999-041
Extra joint for headrest	-	0350-999-043
Headrest stretch cover	-	0350-999-551
Back extender sides and covers also available. Contact Cobi Rehab for more information.		